Gebrauchsanweisung – Operating instructions Gehgestell – Folding Walkers **Gecko**

Art.-Nr. / Art. No. 700 100 100 starr, faltbar / rigid, foldable

Art.-Nr. / Art. No 700 100 200, reziprok, faltbar / reciprocal, foldable

drive
MEDICAL DESIGN & MANUFACTURING

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This product improves or compensates for restricted walking ability (e.g. by increasing the radius of action and/or improving your mobility). Persons suffering from restricted hand/arm mobility need special fixtures and adjustments on handgrips or hand/arm rests.

For persons with disturbed equilibrium: Resting surfaces of the folding walker have to be secured for better balancing.

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Gehgestell, starr –
Folding Walker rigid

Folding Walker, Operating Instructions

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The folding walker is delivered ready for use. Simply unfold it – and move!

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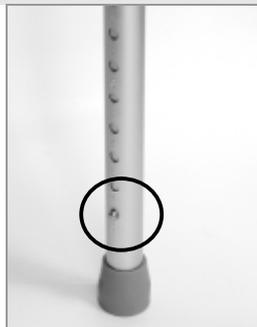
Place the folded walker in front of your body. Hold the hand grips and fold it apart until the locking mechanism of the telescope slide bar catches it. Check the locking mechanism – the slide bar must not move.

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*Press in the spring-loaded buttons of the feet and pull the feet to the desired length. **Make sure to adjust all feet to the same height to prevent the danger of tipping over.***

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When walking, please make sure not to hold the walker too far away from your body. Always hold the walker straight and place it onto its four feet. Make small steps only, and hold on to the hand grips. Make sure to stand firmly before moving the walker to another position. If used diagonally to your walking direction, the walker will tip at an angle of inclination of more than 3,5°.

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Art. Nr. 700 100 100 = 120 kg

Art.-Nr. 700 100 200 = 100 kg

Make sure that the four feet of the walker stand firmly on the ground.

Make sure that the height adjustment is the same for all four feet.

Do not use the product if it is damaged!

Only use the walker on solid and flat surfaces.

Do not use the walker to climb (or similar).

Do not use the walker to carry bags or other pieces of luggage.

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Walking aid is suitable for further use.

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Use commercially available detergents and disinfectants and use a soft cloth.

The folding walker is maintenance-free.

Any repair work to be carried out by a specialist dealer only.

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We issue a warranty of 24 months from date of purchase. Subject to technical alterations.

Stand - As per: 07/2008