Rollator Nitro Carbon

REF.: 724 500 000

CE

Teile des Rollator – Parts of walking device – Pièces du Rollator –Onderdelen van de rollator

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Stand: 01/2015

Indication / Contraindication

This walking aid is suitable for a single user with limited mobility. It is suitable for indoor and outdoor use.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

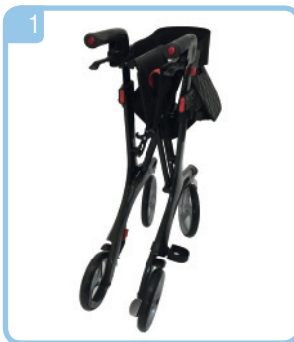
For persons with balance disorders are walking frames not recommended, this is to be checked by an experiment in an individual case. If it is necessary a companion must be present.

Scope of delivery contents

1. Rollator
2. Bag
3. Back rest
4. Operating instructions

Unfold

Position the Rollator on its wheels. Open the Rollator. Now press the both pipes from the seat downwards until they interlock. (see pictures 1 - 2).

Fitting the bag

Fold the Rollator a little and fix the bag with the velcro fasteners into guiding part on the Rollator, unfold the Rollator – ready (see pictures 3 - 4).

Height adjustment handgrips

Press the interlock button for handgrips below and pull the handgrip upwards at the same time.

As soon as the handle moves, release the button then this automatically arrests in the next possible height.

Repeat this process until the needed height is reached. (see picture 5).

Breaking / Parking

Pull both brake levers upwards to apply the brakes whilst the rollator is moving (see picture 6). Always brake slowly and steady.

Press both brake levers down to apply to parking brakes whilst the rollator is stationary (see picture 7). Pull up the brake levers and release to release the parking brakes.

The effect of the brakes can be affected by abrasion of the wheels. **Therefore test regularly the brakes.**

Adjustment of brake

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then turn left or right (picture 8).

Sitting

Position the rollator on an even and solid surface.

Then lock the brakes (parking position).

The rollator must stand and must not slip.

Sit straight up and do not lean backwards (see picture 9).

Folding / Transport / Storage

To Fold, pull seat upwards by the handle on the seat plate. The rollator can now be folded complete and secured by using the red safety bow as shown above. (picture 10 -11)

Safety instructions

Use only as a walking aid!
Use only on even and solid surfaces.

The rollator must not be used for standing on or as a wheelchair! It must not be used for transport of persons or animals.

Maximum user weight: 150 kg

The use on escalator`s is expressly forbidden!

Regularly check the screws and fasteners on the rollator and retighten if necessary.

Use bag only for transport of objects and do not overload. Maximum load: 5 kg

Do not expose the product to temperatures above 40°C for a long time, as this can cause the rollator parts to overheat and burns to the user.

Cleaning / Maintenance

Clean regularly by hand with a damp cloth!
Do not use any aggressive detergents!

Reprocessing

The product is suitable for reprocessing in line with manufacturer's recommendations.

Warranty

We issue a warranty of 24 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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