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Dear users,

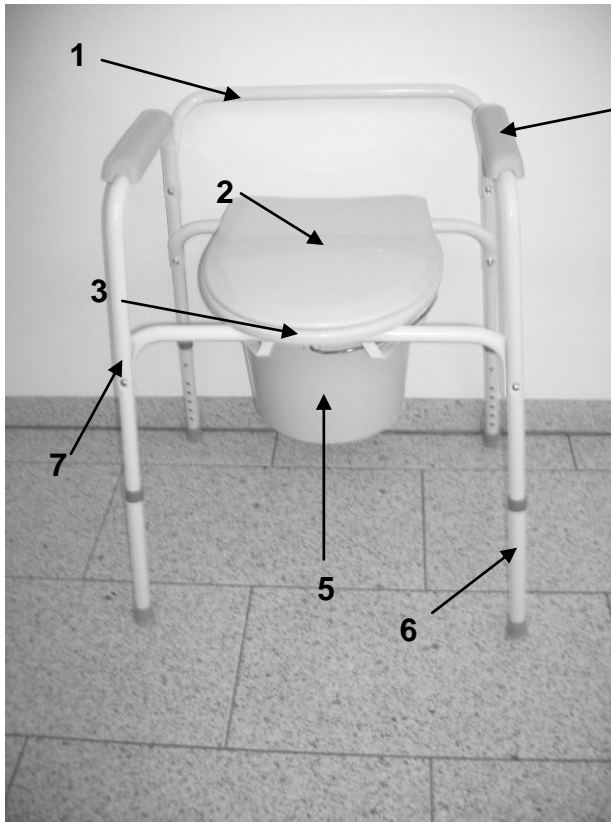
we congratulate on the purchase of a technically mature and to themselves on the newest state of the technology of located product.

This instructions serves you and the care staff as an action basis for sure, safe use of the toilet frame and a sensible use of the product possibilities.

The instructions describes the preparation, service, use, servicing and care of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

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Toilet aids are necessary, with clinical pictures or impediments make the insured person on account of lacking ability impossible to use for getting up/or sit down, customary toilet bowls, also with a raised toilet seat without armrests.

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1. Toilet aid complete assembled
2. Bucket with lid and splash protection
3. Operating instructions

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- *Read this instructions carefully.*
- *Before use all components are to be checked for damages and firm seat.*
- *The steadiness of the toilet aid is high, nevertheless avoid lateral lean out – **Tilting danger!***
- ***Protect from unauthorized use, e.g., by children.***

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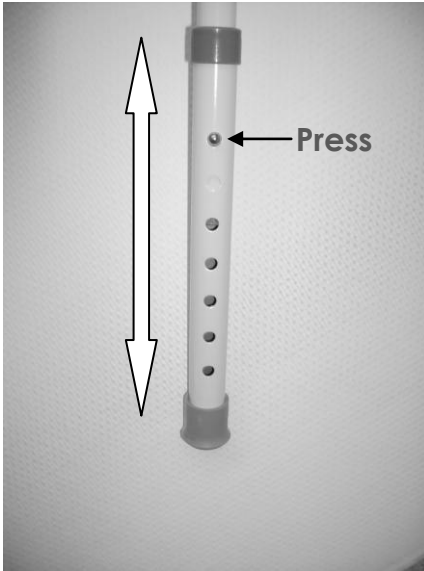
This product is suitable for the reapplication.

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With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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For high adjustment press button and move the standing leg desired

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The commode chair is completely mounted. Insert the bucket with lid as illustrated.

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Instead of the bucket you can also assemble the splash protection similar for use at a customary toilet bowl.

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Clean regularly by hand with a damp cloth and usual detergents!

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Do not use any aggressive detergents!

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Frame: *Welded steel tube, powder coated*

Toilette seat: ABS

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Armrest: PP

Armrest: PP

Bucket, splash protection, lid: ABS

Eimer, Spritzschutz, Deckel: ABS

Typenschild – Type label

The image shows a rectangular type label for the 'Toilettenstützgestell - Toilet aid TSG 130'. On the left side, there are three numbered arrows pointing to specific fields: 1 points to the product name, 2 points to the reference number (REF) field containing '500 800 100', and 3 points to the serial number and date of production (SN / MMY) field containing 'XXX XXX / 29/08'. On the right side, there is a 'drive' logo with the tagline 'MEDICAL DESIGN & MANUFACTURING'. At the bottom right, there is a CE mark with an information icon (i) and an arrow pointing to it, labeled with the number 4. Below the main text, it states 'Max. Belastung - Max.load: 130 kg' and provides the manufacturer's contact information: 'Drive Medical GmbH & Co. KG, 88316 Isny/Allgäu (Germany)'.

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Internet www.drivemedical.de
E-mail info@drivemedical.de

We issue a warranty of 24 months from date of purchase.

*Subject to technical alterations.
Errors and omissions excepted.*

Drive Medical Limited
Ainleys Industrial Estate
Elland
GB WEST YORKSHIRE HX5 9JP

Phone **+44 1422-314488**
Fax **+44 1422-314489**
Internet www.drivemedical.co.uk

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