Indikation – Indication

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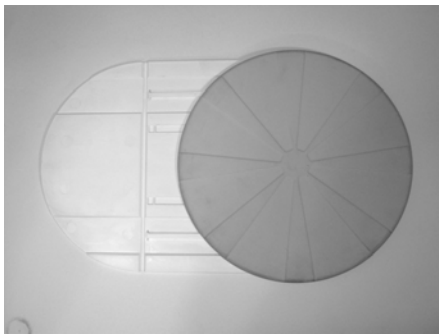
*Considerable movement restrictions and functional restrictions of the pelvic belt and the lower extremities which make the independent rotation of the body around the roll axis impossible or complicate to a great extent.
 The transfer and rotating aid is delivered completely mounted and immediately useable.
Persons with limited sense of balance may not use the product without assistance.*

This product is suitable for reapplication. Check the product before about intactness and follow the cleaning and disinfections tips to clean the product

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The turntable is placed on the bathlift or on the floor. The disabled person sits down or put his/her feet on the disc and is brought, if necessary with support of an auxiliary person, in seated (or standing) position. This occurs under exploitation of the still available forces of the body and upper part of the body of the patient. Now in this seat position or standing position a rotary movement on the disc, and with it a position change, is possible.

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Do not use the unit if it is damaged!
Danger of falling!
Maximum load: 140 kg

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Do not use abrasives!

Clean by hand with a damp cloth! Also possible to be cleaned in a dishwasher.

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For cleaning the turntable can be simply drawn off from the base plate (1.2), this is only clipped in and must be pressed in after the cleaning again (3)

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The product is maintenance-free, but it should be checked regularly for cracks or other damages. In the case of a damage do not use..

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We issue a warranty of 24 months from date of purchase. Subject to technical alterations.

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Drive Medical GmbH & Co. KG · 88316 Isny/Allgäu (Germany)