# **TOPRO TAURUS H**

CE

[www.topro.no](http://www.topro.no)

**Bruksanvisning 2 - 4 NO**

**User manual 5 - 7 EN**

**Bedienungsanleitung 8-10 DE**


**A**

**814846**

**B**

**814001**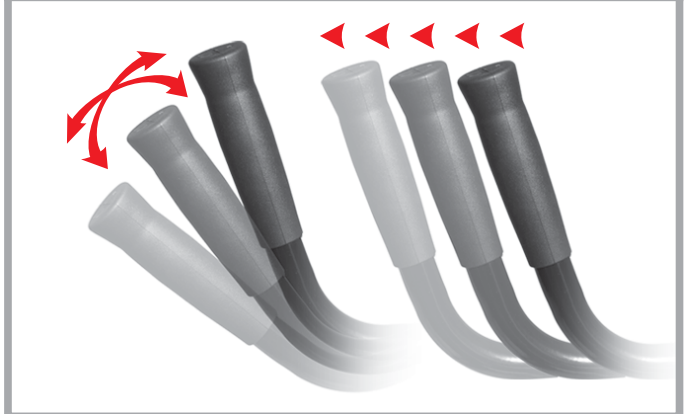


**C**

**814810**

**D**

**814034**

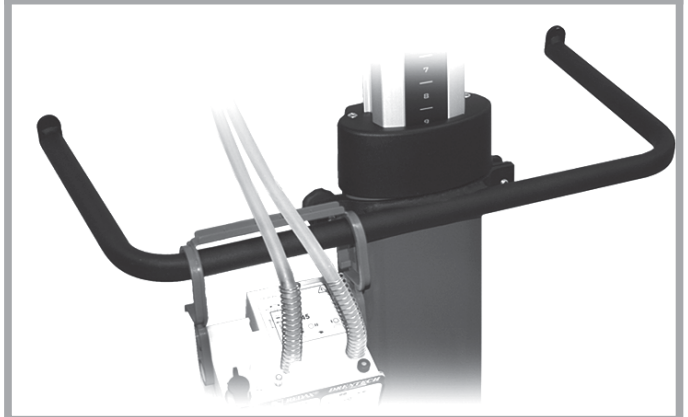**E**

**814821**


**F**

**814820**

**G**

**814645**







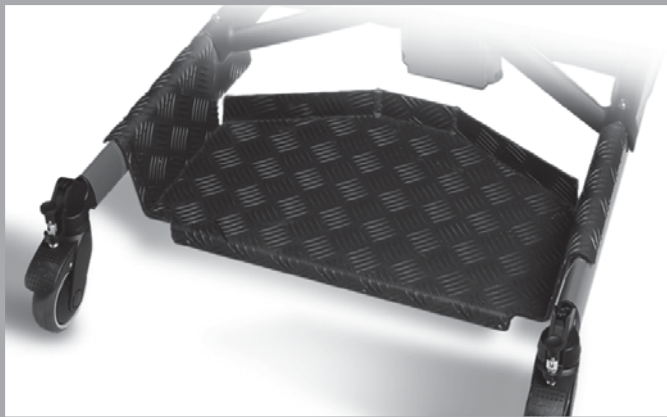


**H**

**814793**

# TOPRO TAURUS H

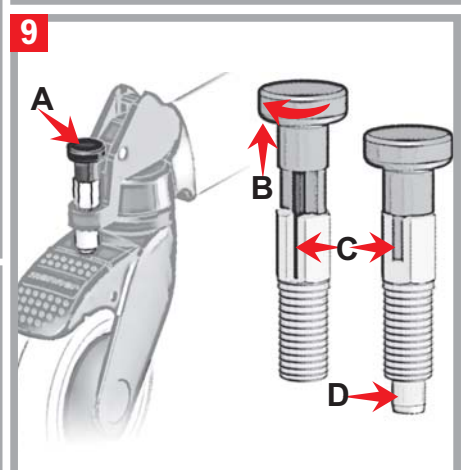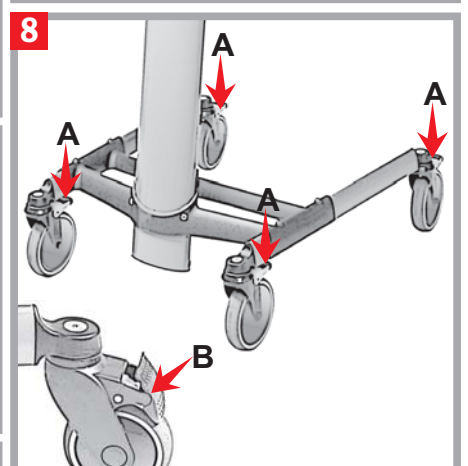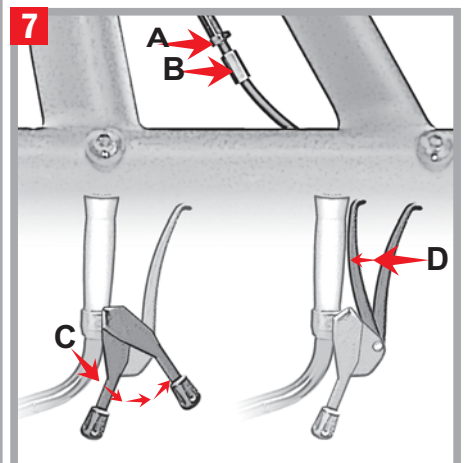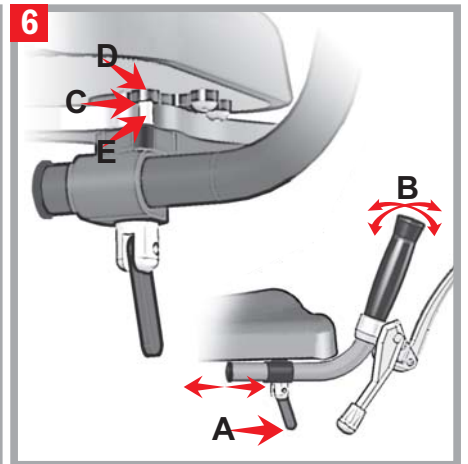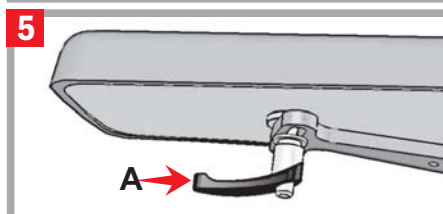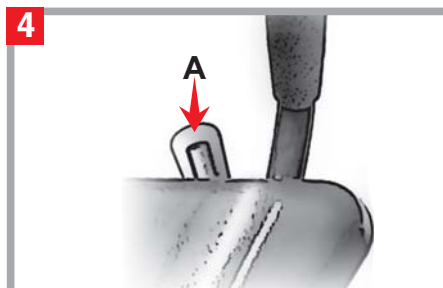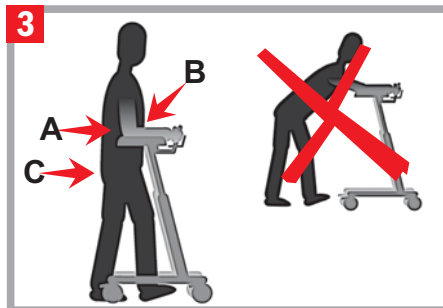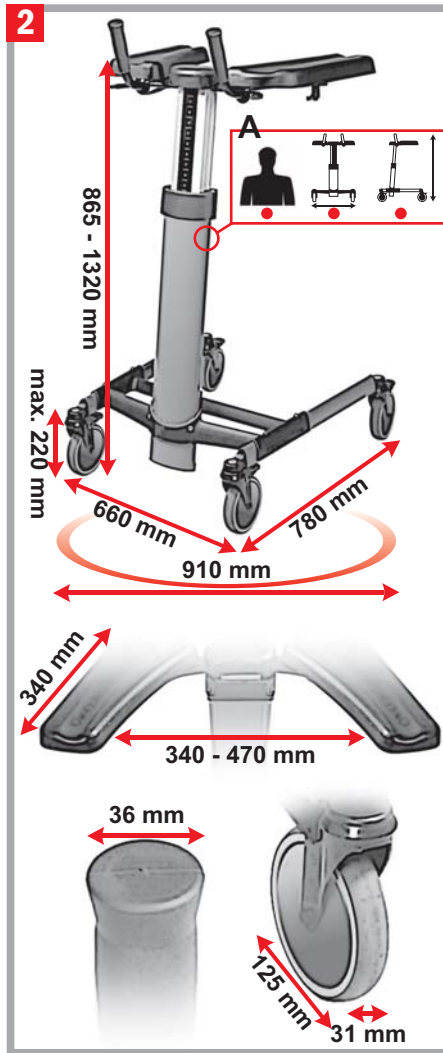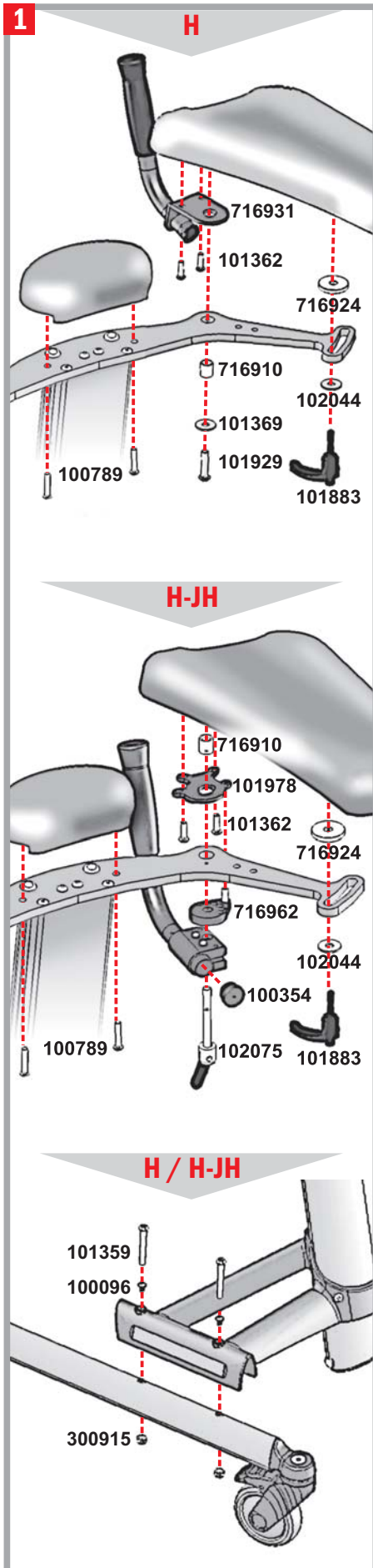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## Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security gives great accuracy and maximum quality in all levels of production. The product is declared to be in conformity with the Medical Device Directive 93/42/EEC.

### Guarantee

TOPRO TAURUS is guaranteed free for faults and defects for 2 years. An exception is if the damage is caused by incorrect use or parts that are exposed to natural wear and tear (such as e.g. brakes, wheels and handles). For repairs in the guarantee period, please refer to your local mobility shop or dealer.

**The red numbered squares below refer to the numbered illustrations on the inside of the cover.**

## 1 Assembling and adjusting

TOPRO TAURUS H and H-JB need assembling. TOPRO TAURUS H-JHB is delivered completely assembled. Adjust the forearm supports and handles, see figure 5.

## 2 Measurements

A The label on the rear side of the column shows the product name/model and

- Maximum weight of user
- Maximum width of walker
- Maximum height of walker

Technical data	
Distance between front and rear wheel	780 mm
Width between front wheels	660 mm
Height of forearm supports	865 – 1320 mm
Distance between forearm supports	340 – 470 mm
Turning width	910 mm
Hand grip length	90 mm
Hand grip width	36 mm
Grip distance driving brake	74 mm
Wheel width	31 mm
Wheel diameter	125 mm
Weight walking table (w/o accessories)	18 kg
Maximum weight of user	150 kg
Material	
Frame	Aluminium
Push handles	PVC
Brake handles	PA6
Forearm supports	PU-foam

### Models

The TOPRO TAURUS H is delivered as standard edition, plus two variants with the following supplementary equipment:

**JH** Adjustable handles.

**JHB** Adjustable handles. Driving and parking brakes. Swing lock (directional block).

	TOPRO art. no.
<b>TOPRO TAURUS H</b>	<b>814785</b>
<b>TOPRO TAURUS H-JH</b>	<b>814786</b>
<b>TOPRO TAURUS H-JHB</b>	<b>814787</b>

### Intended use

The TOPRO TAURUS is designed and approved to be used indoors on plain foundation. The walker safely supports users with poor balance and/or poor mobility.

### **3 Important to know about safety and use**

- The walker must be adjusted to the recommended height for right support and safe walking: Stand with relaxed shoulders and have your elbow at a 90 degree angle **A**. Hold the walker as close to your body to maintain the correct support and a safe walk **B**. The easiest way to keep your balance is to walk straight, look forward and hold your backside in **C**.
- Do not push the walker too far from your body as this can cause unnecessary strain and you could fall.
- Do not push the walker forward if the front wheels have stopped at an obstacle. The walker may tilt and cause an accident.
- Max user weight is 150 kg. Max user height 2 m.
- Do not let children play with the walker.
- Some parts of the walker might feel hot or cold when exposed to extreme temperatures.

### **4 Height adjustment**

#### **Hydraulic height adjustment.**

Pull the handle up **A**. Use a little body weight to press down or reduce the counter weight to increase the height. Release the handle at the required height.

### **5 Adjusting the forearm support**

The forearm supports are adjusted individually. Unscrew the lock underneath the forearm support **A** until the cushion is moveable. Place at the required position and tighten the screw. **NB** If the lock sticks out, it can be dangerous. Pull the lever down and rotate it so that it doesn't stick out.

### **6 Angle adjustment and twisting the handle (TOPRO TAURUS H-JH and H-JHB)**

Unscrew the screw **A** until you can move the handle. Adjust the handle **B** to the required angle, length and twist and tighten the screw. **NB** The screw must not be screwed too far out as the parts will fall from each other. If this happens, then put the pin **C** in the hole **D** and **E**. It is not possible to pull the handles all the way out; there are physical stops both ways.

See figure 1 for more detailed instructions.

### **7 Brakes**

#### **(TOPRO TAURUS H-JHB)**

##### **Adjusting the brakes.**

Unscrew the nut **A**. Tighten / loosen the brake wire with the adjusting screw **B**. Make sure the brakes work properly and don't sweep when brakes are off. Adjust both sides.

##### **Using the parking brakes.**

The brake lever **C** works on both rear wheels simultaneously. Push the brake lever forward. It has some resistance before it stops and locks both the rear wheels. Pull the brake lever backwards to release the rear wheels. Inspect the brakes frequently to make sure they work properly.

##### **Using the driving brakes.**

The brake lever **D** works on both rear wheels simultaneously. Pull the brake lever towards you to stop the rear wheels. When the brake lever is released, the rear wheels are released. Inspect the brakes frequently by following the information above.

### **8 Using the foot brakes**

Step on the foot brakes **A** on the rear wheels or when required on all wheels until you hear a clicking sound. The wheels are now in a locked position **B**. To release the wheels, step on the back of the foot brake pedal so that it comes up again.

### **9 Locking the swing (directional block) (TOPRO TAURUS H-JHB)**

To be done with an index pin **A** on both wheels. Make sure the mounting bracket for the foot brake is pointing backwards.

### This is how the index pin works:

Locking: Twist the upper part of the index pin **B** so that it fits into the track of the lower part **C**. By doing this the pin **D** will slide out and lock.

Releasing: Pull the upper part of the index pin **B** up and turn it 90 degrees. It will then lock in the upper position. The pin **D** slides in and the index pin is unlocked.

### Material / recycling

The walker is made of plastic coated aluminium pipe, plastic couplings and material of plastic and polyester. The forearm cushion is of polyurethane. Most parts can be recycled.

### Cleaning

The walker can be cleaned with household detergents. The walker can be hosed.

### Cleaning with disinfections must be left to experienced personnel

To disinfect the walker, there must be used disinfection containing 70 - 80 % Ethanol. We advise against the use of disinfections containing chloride and phenol. The manufacturer cannot be incriminated if damages occur under disinfecting caused by wrong use or when cleaned by unauthorized personnel.

### Inspection / Maintenance / Re-use

The wheel bearings are sealed and maintenance-free. All the other parts do not need to be lubricated. It is recommended that general inspection and maintenance is carried out and that the items below are checked frequently: tubes and frame, handles and forearm supports, brakes, wheels and accessories. This also applies when the walker is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of the brakes and cleaning and disinfection of the walker.

### When something does not work

1 = possible reason.

2 = action.

#### Brakes

1 Worn out / defect wheel or brake.

2 Change the complete wheel.

1 Unadjusted brake.

2 Adjust the brake wire.

1 Worn out / defect brake mechanism on wheel.

2 Change the brake mechanism.

1 Worn out / defect brake handle.

2 Change the complete brake handle.

1 Worn out / damaged brake wire.

2 Change the brake wire.

#### Height adjustment

1 The gas spring is worn out or damaged.

2 Change the gas spring.

1 The control arm is maladjusted.

2 Adjust the control arm.

### Accessories

For pictures and TOPRO article numbers see the inside of the cover. Accessories mounted on the walker may influence stability, we therefore advise you to use them carefully.

**A** Brake handle left side

**E** Small basket

**B** Adjustable handles

**F** Holder for oxygen

**C** Table

**G** Footplate

**D** Stand for body fluid-boxes

**H** Infusion stand

The accessories are to be ordered separately to give the possibility to compose a TOPRO TAURUS that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage [www.topro.no](http://www.topro.no)

**In case of faults or questions please contact your mobility shop or dealer.**

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