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# Rollator vital plus 54

Art.-Nr. 11448 501      grau  
 Art.-Nr. 11448 511      mokka  
 Art.-Nr. 11448 541      eisblau

# Rollator vital plus 62

Art.-Nr. 11448 500      grau  
 Art.-Nr. 11448 510      mokka  
 Art.-Nr. 11448 540      eisblau

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**Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this product is to be used by other persons, they must be fully informed of these safety warnings before use.**

### Indication/Contraindication

The rollator is a mobility aid for people with mobility restrictions, e.g. people with a disability that makes walking difficult or who cannot walk for long, providing them with the opportunity to go further afield.

Staying on the move with a rollator not only serves to improve circulation, coordination and balance, but it also enhances the user's well-being.

This rollator is designed both for use in enclosed areas and in the open. It is equipped with a removable bag.

The rollator provides a seat for short rests. It can be folded easily for transportation, thus saving space. The rollator should not be used by persons with perception impairments, serious problems with their balance or an inability to sit. The user must have adequate perception skills and adequate physical ability to be able to steer a rollator.


### ⚠ Safety Indications ⚠

- Inspect the product for possible transport damage before first use, paying particular attention to the screw and plastic connections as well as the brake cables and making sure the wheels and wheel suspension are intact. A damaged rollator must not be used.
- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Check that the Rollator is in perfect condition every time before you use it.  
Check the following components and make sure:
  - all screws, nuts and other parts are firmly tightened
  - all wheel surfaces are intact
  - the height adjustment screws are firmly tightened
  - the brakes function both as parking brakes and hand brakes
  - there are no visible signs of wear on the frames, welded joints and plastic elements
  - wheels and wheel forks should be free of dirt and turn smoothly.
 Do not use a damaged or defective Rollator!
- Ensure that the Rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the Rollator.
- Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The Rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these dangers.
- On smooth surfaces such as tiles, stone flooring, PVC, etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your Rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the Rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the Rollator cannot slide away.
- The Rollator should not be used as a support when getting up from a crouching or sitting position! It should only be used for its intended purpose and should not be used for transporting objects over 5 kg or persons.
- Ensure that the handles are adjusted to the correct height for the user in order to avoid improper weight load or falls. We recommend you walk between the handles. Always hold both handles; otherwise you can lose your balance.
- Make sure that your hands or clothes do not come into contact with the moving parts of the Rollator! Be particularly careful when folding and unfolding the Rollator.
- The Rollator should be used on the pavement. The Rollator should not be used in moving traffic. For your personal safety on the roads it is recommended that you also wear your own reflectors as well as those fixed to the Rollator.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brakes on so that it cannot fall over or roll away. Look for a seat close by.
- Be careful near children. Children should not play with the Rollator!
- The maximum permissible loading for the Rollator is 150 kg. This includes the maximum load in the bag of 5 kg.
- Do not hang any handbags or carrier bags on the handles, the balance of the Rollator can be impaired if you do.
- Do not carry out any adjustments or modifications, which impair the safety of the design.
- Only use accessories approved by the manufacturer. Please also read the instructions for use for each accessory.
- Do not carry or pull the Rollator either by the strap to fold it up or by the brake cables.
- In strong sunlight parts of the rollator, e.g. the handles and seat, can get hot. Please ensure that you cover your rollator up or leave it in a shady place.
- Please prevent skin contact with the Rollator if the Rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.
- Please prevent running into door frames, furniture, etc. with the Rollator and check the secure position of the Rollator when parked. The frame, connection elements and welding seams may be damaged if the Rollator tips over.
- Please also prevent placing stress on the Rollator when folded together! This could damage the frame, connection elements and welding seams.
- Do not use the Rollator in stairs or escalators.
- If you have any queries, please refer to your dealer.

## Contents/Description of the parts:

- |                            |                |                                   |                           |
|----------------------------|----------------|-----------------------------------|---------------------------|
| A. Handle                  | E. Brake cable | I. Strap for folding the Rollator | L. Bag holder             |
| B. Break lever             | F. Side rails  | J. Brake pads and edge guide      | M. Magnetic lock          |
| C. Seat                    | G. Frame       | K. Bag                            | N. Stick Holder           |
| D. Height adjustment screw | H. Step aid    |                                   | O. front wheel, steerable |
|                            |                |                                   | P. back wheel with brake  |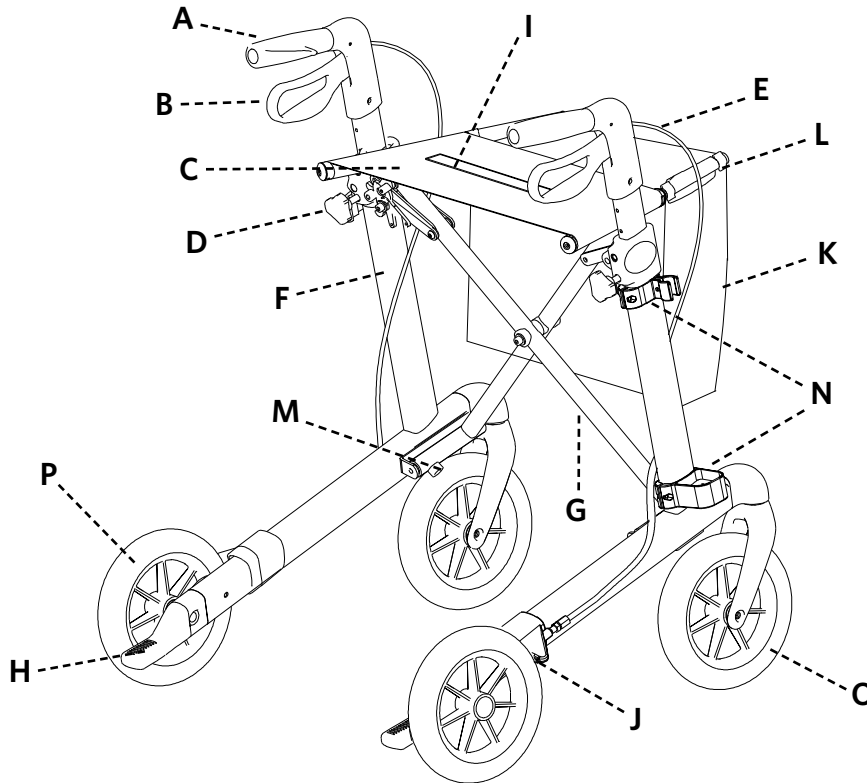

## Assembly

Remove all packaging. The packaging should be disposed of responsibly.

### Opening the Rollator

1. Pull the handles apart (pic. 1).
2. Push the side rails downwards until they engage. The Rollator is now fully opened (pic. 2).
3. Make sure the Rollator is engaged before you use it.

pic. 1

pic. 2

### Folding up the Rollator

1. Fold the Rollator up by pulling the strap upwards (pic. 3).
2. Push the handles inwards and the Rollator together (pic. 4). The fully folded Rollator is locked with magnets (pic. 5).
3. When folded, the rollator can easily be carried or pulled by the basket holder (pic. 6). Do not carry or pull the rollator by the brake cables or the strap in the seat.

pic. 3

pic. 4

pic. 5


pic. 6

### Height adjustment of the handles (pic. 7)

Adjust the height of the handles so that the user can walk upright, without bending over. It is recommended that the user stands upright with arms at their sides between the handles. The grips should be adjusted to the height of their wrists.

1. Loosen the height adjustment screw D.
2. Move the handle to the desired height.
3. Tighten the height adjustment screw D.

Adjust both handles in the same height. The printed numbers on the inside of the push handles help you easily adjust your personal grip level again (pic. 8).

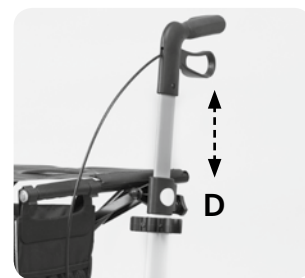pic. 7

pic. 8

### Brake (pic. 9)

Check the brakes function before use.

1. Speed is reduced by pulling the brake lever upwards.
2. The parking brake is activated by pressing the brake lever down until it engages.
3. The parking brake is released from its locked position by pulling the brake lever upwards.
4. The Rollator should not be pushed along with the parking brake on. If the Rollator is pushed along with the parking brake on this can cause extreme wear to the tyres and brakes and their function impaired.
5. The Rollator must not be used with damaged brakes.


pic. 9

### Brake adjustment (pic. 10)

The brake shoe can be adjusted if the braking force lessens as a result of wear on the wheels. The brakes must be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be about 2-3,5 mm.

1. Loosen the locking nut.
2. Adjust the braking force by turning the brake cable spanner inwards or outwards.
3. Check the braking function by turning the wheels and operating the brake levers. The wheels should turn freely but stop immediately when the brakes are applied. When the brake is activated, the wheels shall not be possible to move.
4. Tighten the locking nut firmly again.

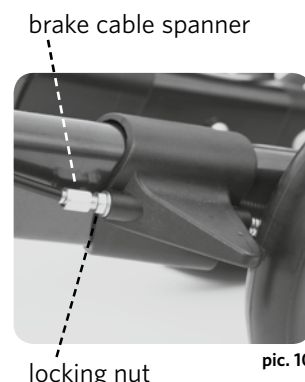pic. 10

### Seat

Activate the parking brake before you sit on the seat. Always sit fully on the seat with your back in the direction of travel. (pic. 11) Do not lean against the seat! Do not lean over the sides or backwards while sitting on the rollator. That could impair the stability of the Rollator. Do not use the seat on a sloping surface. Ensure that the front wheels are parallel to the direction of walking.

The seat has been tested for a user weight of up to 150 kg.

pic. 11

### Negotiating obstacles with the support of the step aid

Tread on the step aid in order to raise the front wheels of the Rollator so that you can negotiate a doorstep, kerb or another obstacle more easily (pic. 12).

pic. 12


## Stick Holder

You can use the stick holder to attach your walking stick simply and securely to the rollator and always have it handy for when you need it. You can either fix the stick holder to the right or left hand frame of your rollator. Unscrew the fixing screws with the tool supplied and place the small basket at the lower end around the vertical frame (pic. 14). Ensure that the basket points outwards. Insert the fixing screw through the opening provided (pic. 15). Tighten the screw so that the stick holder cannot become loose (pic. 16).


Proceed with fixing the upper bracket in exactly the same way. It is fixed at the upper end of the vertical rod of the frame. Keep the tool in a safe place in case you wish to reposition or remove the stick holder. When you wish to place your stick into the stick holder or remove it, apply the brakes of your rollator firmly so that it cannot roll away. To fix your stick, first place the rubber pad into the lower basket of the stick holder. Then press the shaft of your stick into the upper clip of the stick holder until it is fixed. To remove the stick, pull it firmly out of the clip. An attached stick can project from your rollator. Please be extra cautious so you do not catch the stick on anything or injure other people with it.


pic. 13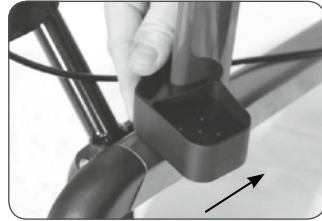

pic. 14

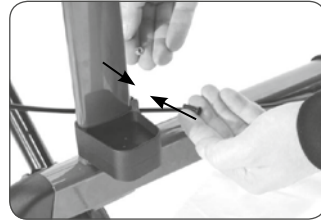pic. 15

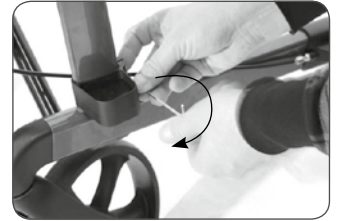pic. 16

## Care and Maintenance


### Rollator

The Rollator may only be used as a walking aid. Keep the Rollator free from dirt, so that it remains functional. The Rollator can be cleaned with a neutral cleaning agent and a cloth and then rinsed and dried. Dry the Rollator thoroughly after cleaning or exposure to rain. The brake pads and springs should also be cleaned regularly.

The Rollator should only be disinfected by a specialist. The product can be disinfected with a disinfectant with 70–80 percent ethanol content. Disinfectants containing chlorine or phenol should not be used. Do not use caustic solvents.

Do not use the Rollator in temperatures below  $-35^{\circ}\text{C}$  or above  $+60^{\circ}\text{C}$ . Apart from small corrections on the paintwork, changing wheels or adjusting the brakes all repairs should be carried out by a specialist retailer.

Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

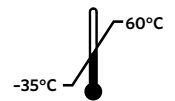### Bag

Small items can be carried conveniently in the bag. The inside pockets are suitable for storing items like keys or your purse or wallet. The bag is designed to carry up to 5 kg. It can be hung up and removed again easily using the integrated handles (pic. 17+18).

pic. 17

pic. 18

## Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

## Storage

Store the Rollator light and dust-protected in a dry place. Do not store near a heat source. ☂️ ⚡

## Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

## Technical Details

Material:

- Frame: Aluminium with plastic-segments
- Wheels: PU

### Rollator vital plus 62

- Dimensions: 60 x 66 x 74-101,5 cm (W x L x H)
- Folding measure: 22 x 66 x 74-101,5 cm
- Wheels: 20,2 x 3,5 cm
- Seat level: 62 cm
- Distance between handles: 44 cm
- Weight: 7,12 kg + bag (380 g)
- Turning circle: 84,5 cm
- Recommended body size: 170-210 cm
- Max. loading capacity: 150 kg

### Bag

- Material: 100 % Polyester
- Dimensions: 32 x 15 x 28 cm (W x L x H)
- Permissible weight: max. 5 kg

### Rollator vital plus 54

- Dimensions: 60 x 66 x 66-85 cm (W x L x H)
- Folding measure: 22 x 66 x 66-85 cm
- Wheels: 20,2 x 3,5 cm
- Seat level: 54 cm
- Distance between handles: 44 cm
- Weight: 6,96 kg + bag (380 g)
- Turning circle: 84,5 cm
- Recommended body size: 150-180 cm
- Max. loading capacity: 150 kg

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