PFLEGEROLLSTUHL DORIS

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Dear User,

Thank you for choosing this product. This user manual describes how to prepare, operate, use, maintain and care for the product. It is a guideline for users and caregivers to ensure the safe and risk-free use of the Doris Relax Chair. Please read this user manual carefully and keep it readily at hand at all times.

Intended Purpose

The Doris Relax Chair is a therapeutic wheelchair designed for comfort and relaxation that fulfils a variety of patients' needs. It can be used for patients with mental or physical disabilities, provides relief for patients suffering from geriatric infirmity or muscular disorders, helps Parkinson patients relax, calms down Alzheimer patients, helps prevent pressure ulcers, makes sitting easier for patients with arthritis, gives peace of mind to people with limited mobility, helps patients suffering from anxiety or depression to assume an embryonic posture and gives them a feeling of warmth and emotional comfort, it is helpful in stroke patients' regeneration process and protects Chorea Huntington patients suffering from akathisia.

EMOTIONAL COMFORT: The Doris therapeutic relax chair provides the user with a feeling of warmth and protection, which helps patients relax and increases their well-being.

MOBILITY: The Doris Relax Chair can be used for a wide range of purposes. Its castors make it extremely manoeuvrable, and the sitting or the lying position in the chair are quick and easy to adapt and continuously adjustable.

SAFETY: The pleasantly moulded shape of the chair adapts to the body and provides the user with a feeling of support, warmth and security.

COMFORT: The perfect shape and the top-quality upholstery that redistribute pressure make the chair perfectly comfortable when sitting or lying it. Easy access makes patient transfer easy. The Doris Relax Chair is certainly not a chair like any other – it is manufactured mainly by hand with a great deal of technical expertise.

TECHNICAL SPECIFICATIONS

W 65 x L 115 cm, Height of backrest: 74 cm

Length at lowest backrest setting: 142 cm

Seat width: 43.5 cm

Seat height: 57–72 cm

Seat depth: 53 cm

Weight: 45 kg

Max. load/supported weight: 150 kg

Frame material: Coated steel

Upholstery material: Special foam

Cover: Synthetic leather in blue, red, yellow or green

Two different models of the Doris Relax Chair are available:

For indoor use

4 castors (Ø 12.5 cm), rear castors with brakes

For outdoor use

2 castors (Ø 12.5 cm) and two rear wheels with brakes (Ø 40 cm)

SAFETY INSTRUCTIONS

- Only use the Doris Relax Chair if you can ensure that a trained assistant or caregiver is present at all times during use.
- Only patients whose body height and physical build enable them to adopt the seated and lying position in the chair should use it.
- The Doris Relax Chair is suitable for indoor and outdoor use.
- The Doris Relax Chair may only be used or operated by people who are familiar with the instructions in this manual. Always ensure that there are no obstacles within the operating range of the chair such as steps, staircases or objects on the floor, which could cause the relax chair to tip.
- Do not perform any operations or movements unless you are absolutely sure of their outcome; if in doubt, contact the next service point or contact the manufacturer directly. Only trained staff should provide instructions for using the Doris Relax Chair. Watch out for potential risks! Use of the Doris Relax Chair and its accessories require medical supervision.

- Never use the Doris Relax Chair on uneven surfaces, steep slopes or stairs. Prohibit unauthorised use of the Doris Relax Chair. Always apply brakes when the chair is not being moved, even on flat ground. Before adjusting any part of the chair, always apply the brakes. Always check functionality of the brakes, inspect castors for wear and tear, ensure that all accessories and parts that may become loose are firmly attached.
- Never park the Doris Relax Chair on a slope.
- Never use the Doris Relax Chair if parts are damaged or missing. Ensure that all adjustable parts are locked in place before each use of the Doris Relax Chair. Clean the Doris Relax Chair as described in the "Cleaning" section of this manual. Contact your specialist dealer who supplied you with the Doris Relax Chair in the event of an accident or defect. Please keep this user manual readily at hand at all times for future reference.
- The Doris Relax Chair requires annual servicing to ensure its longevity and prevent damages to the product or harm to its user. Maintenance services must be performed by an authorised person in accordance with the maintenance schedule.

ADJUSTMENT AND OPERATION

Bring the chair into the upright position by pushing down on the tilt adjustment lever **4** before the user sits down in it. Position the chair close to the patient to ensure easy access. Ensure that the legrest **7** is positioned underneath the seat **3**. The legrest is folded away by applying the adjustment levers to the left and right of the legrest hinges. To adjust the footrest, release both levers. Reapply the lever when you have reached the desired position. As soon as the patient is sitting down in the seat, you can adjust the seat to sitting or lying position suited to the individual patient's needs by applying the tilt adjustment lever. The backrest and seat angles will automatically stay in an orthopaedically and anatomically correct position, regardless of the backrest tilt. By releasing the legrest adjustment levers to the left and right of the hinge, you can now position the legrest in the desired height and tilt angle. Once you have reached the desired position, the hinges are fixed in place using the two levers.

OVERVIEW

- 1 Headrest
- 2 Lateral head support
- 3 Ergonomic seat shell
- 4 Push bar and steering bar with tilt inclination lever
- 5 Lateral body support
- 6 Rounded, supporting wedge cushion
- 7 Legrest, height and tilt adjustable
- 8 Gas pressure spring to adjust seat tilt
- 9 Sturdy coated steel frame
- 10 Castor
- 11 Castor with brake

ADJUSTMENT

Operating the Doris Relax Chair is easy and safe, however, a thorough introduction to its features is required and should be provided by the specialist dealer or therapist who has supplied the chair. It is essential to always follow the instructions contained in this user manual.

FIRST USE

The Doris Relax Chair is delivered fully pre-assembled. It consists of these components:

- Undercarriage with castors/wheels and brakes
- seat shell with headrest, wedge cushion and legrest

CLEANING AND DISINFECTION

Cleaning the seat: Use a damp cloth and mild detergent to wipe down, then rinse with clear water. Do not use corrosive or harsh detergents to clean or disinfect the chair, or any other detergents that are not suitable for synthetic leathers. If required, disinfect with commercially available surface disinfectants. Please note the safety information and hazard warnings on any detergent or disinfectant you may use and adhere to the instructions.

Cleaning the castors/wheels: always keep castors clean. Regularly remove any dust, hair, lint or other debris that collects in the castors.

MAINTENANCE AND SERVICE

The Doris Relax Chair requires annual servicing, which must be carried out by a person authorised by the manufacturer or its representative. Maintenance service must be carried out in accordance with the maintenance schedule. Never perform maintenance work or exchange parts of the chair or accessories while the chair is occupied! The manufacturer expressly excludes any product liability in such an event.

Like any other technical product, your Relax Chair also requires regular maintenance. The following section describes the steps you should take to ensure that you can continue to benefit from your Relax Chair's features for a long time.

Before moving the chair

Inspect the castors and wheels for visible damage and/or soiling. Remove any dirt as this may affect the functionality of the brakes. If a castor or wheel is damaged, contact an authorised workshop for its repair.

Every 6 months

Depending on the frequency of use, please check:

- Cleanliness
- General condition
- Functionality of castors/wheels
- If rolling resistance is high, clean the castor wheel bearings. If this does not solve the problem, please contact your specialist dealer.

WARRANTY

The statutory warranty period of two years from date of purchase applies. Only original spare parts and accessories may be used. The warranty expires if third-party parts are used or if repairs or modifications of the Doris Relax Chair or its components are carried out by unauthorised persons. The manufacturer expressly excludes any product liability in such an event. The manufacturer also excludes liability for damage to individuals, animals or objects in the following events: improper use of the Doris Relax Chair; use by unauthorised or untrained individuals; improper mounting of accessories or other parts that are not suitable for the Relax Chair; modifications or interventions that were not previously agreed with the manufacturer; use of third-party spare parts and disregarding the instructions contained in this user manual.

DISPOSAL

Please dispose of the Relax Chair in an environmentally friendly manner via an approved waste disposal company or your local waste disposal facility. Please make sure to comply with any applicable regulations

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CARELINE GmbH & Co. KG
CARELINE B.V.