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**MPV**MEDICAL

# MicroDrop<sup>®</sup> Family

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## We hope you get better soon!

### Intended Use

The nebulizer is used for the nebulization of medication (inhalation solutions), which is suitable for compressor inhalators. The *Jet* nebulizer together with the used MicroDrop® compressor and, if necessary, the used accessories form some highly effective devices for inhalation therapy of the lower respiratory airways with medication aerosols. Please clean the nebulizer before you use it for the first time.

The *Jet* nebulizer is not intended as a life sustaining or life-supporting device.

### Important

**The instruction manual is part of the nebulizer.  
Please carefully read the manual before use.**

**Please follow your doctor's instructions!  
Only use medication prescribed by your doctor!  
Do not use any ethereal oils!**

**Do not use any ethereal oils!**

## Precautions

Use the nebulizer and its accessories only for the intended use as mentioned in the instruction manual and follow your doctor's instructions. All other kind of use is misuse of the product and therefore is dangerous. Thus, the manufacturer is not liable for damages caused by wrong, incorrect and unreasonable use.

You must conscientiously follow the general hygiene steps on using your nebulizer, e.g. thorough washing of hands or disinfection if required.

You must not leave your nebulizer with children without any supervision. Some parts of the nebulizer are very small and might be swallowed by children. Therefore, never let your children use the nebulizer without supervision.

Use the "nosepiece" accessory only if expressly indicated by your doctor and paying attention NEVER to introduce inside the nose the nasal bifurcation, but only bring it as close as possible.

## Preparation for Use

**Before each use, as well as after each long break in use, please follow the hygiene steps and ensure that the nebulizer was cleaned, disinfected and, if necessary, sterilized after the last treatment. See chapter "Hygiene". After unpacking your nebulizer and its accessories, please check the completeness**

1. Open the nebulizer by twisting the upper part anti-clockwise (pic. A).
2. Make sure that the nozzle coat (cone) in the container fits well on the cone for the air supply (pic. B).
3. Fill the amount of medication prescribed by your doctor the nebulizer cup (pic. C). The scale on the container helps you to dose the medication liquid. Please take care not to exceed the maximum filling capacity (6 ml).

Close the nebulizer by attaching the upper part and twisting it clockwise. Take care that the nebulizer is closed tightly.

First put Inhalation valve (4) into the nebulizer (1) and afterwards put in the angle piece (3) and the exhalation valve (5). Please put the mouthpiece (2) directly on the angle piece (3) (Fig. E).

When using the mask, see chapter "Use with Mask".

Keep the nebulizer in an upright position and connect the tubing from downside to the nebulizer (pic. D).

Connect the other part of the compressed air tube to the compressor.

After finished treatment, please clean the nebulizer and the accessory as described in the chapter "Hygiene".

## Use with Mask

### General information

Inhalation with the mouth piece is the most effective form of inhalation, since it results a minimum loss of medication on the way to the lung. The masks for children and adults are only recommended for patients who are unable to inhale with the mouth piece.

1. Remove mouth piece (2) and the angle piece (3) of the nebulizer, i.e. pull it slightly and remove it to the front (pic. E).
2. When using a mask the inhalation valve (4) is not necessary. Remove inhalation valve (4) by pulling it upwards from the nebulizer.
3. Finally connect the mask directly to the nebulizer (pic. F).  
Now, you can start with the inhalation.

## Inhalation

**Make sure that all parts are firmly connected.**

**Make sure that the tubing is connected firmly to the air outlet of the nebulizer and the compressor. The medication nebulization can be adversely affected by an incorrectly assembled nebulizer.**

1. Sit in an upright position and relax. Switch on the compressor.
2. Take the mouth piece between your teeth and surround it with your lips resp. hold the mask firmly over your mouth and nose.
3. Breathe in as gently as possible through the mouth piece or mask.
4. Breathe out gently through the mouth piece or mask. The exhaled air passes through the expiratory valve in the mouth piece (valve opens automatically) resp. through the apertures in the mask.
5. Repeat inhalation and exhalation as described until a change in the tone of the nebulizer signals that the nebulization has finished.
6. Then check whether a clearly visible aerosol mist is still flowing out of the mouth piece resp. mask.
7. Stop treatment as soon as the aerosol emerges irregularly.

## Hygiene

### General

**In order to prevent a risk to health, e.g. an infection occurring through a contaminated nebulizer, it is imperative that the following hygiene regulations are followed.**

The nebulizer is designed for multiple uses. Please be aware that there are different hygiene requirements for different fields of application.

- **at home** (single patient use):  
The nebulizer must be cleaned after each use and disinfected once a week.
- **hospital / doctor's practice** (possible change in patient):  
The nebulizer must be cleaned, disinfected and sterilized after each use.
- Additional requirements which regard to necessary hygienic preparations (hand care, handling medication and / or inhalation solutions) in the case of high-risk groups (e.g. patients with cystic fibrosis) can be obtained from the respective self-help groups.

Use of the stated cleaning alternatives is done on the responsibility of the user.

- ▶ Please ensure thorough drying after each cleaning, disinfection and / or sterilization. Damp deposits or residual wetness can offer an increased risk through the growth of bacteria.

- ▶ Please check the parts of your nebulizer regularly and replace any defective (broken, malformed, discoloured) parts. Replace the whole nebulizer at least once a year.
- ▶ Please read and follow the hygiene instructions in the instruction manual of the compressor.

## Preparation

Directly after each treatment, all parts of the nebulizer must be cleaned of medication residue and contamination. For this purpose dismantle the nebulizer into its individual parts (pic. E).

Pull the tube from the nebulizer and compressor. Remove the mouth piece (2) from the nebulizer. Remove the exhalation valve (5) by pulling the air supply completely with the angle piece (3) upward from the nebulizer (pic. F).

Twist the upper part of the nebulizer anti-clockwise and remove it from the lower part. Take the nozzle coat out of the medication cup.

For hygiene procedures at home please read chapter "At home". For hygiene procedures in the hospital, doctor's practice or old people's home please read chapter "Hospital, doctor's practice and old people's home".

## At home

### Cleaning (after each use)

#### **Recommended method: using warm tap water**

Clean all parts of the nebulizer and the mask thoroughly for 5 minutes with warm tap water (max. 60°C). Eventually use a little dishwashing liquid (dosage according to the manufacturer's recommendations). Then rinse all parts of the nebulizer thoroughly under running warm water (for at least 1 min.) without dishwashing liquid. Finally dry all parts as described in chapter "Drying".

#### **Possible alternative method: in the dishwasher**

Select a program with a minimum temperature of 50°C. Distribute the parts of your nebulizer in the cutlery basket.

**Attention: Never together with soiled tableware!**

### Disinfection (once a week)

Disinfect the dismantled nebulizer after cleaning.

#### **Recommended method: boiling water**

Put all parts of the nebulizer (apart from PVC masks) in a pot of boiling water for about 10 min. Make sure the waterlevel in the pot is high enough to keep the parts from toughing the bottom.

#### **Possible alternative: using a steam disinfectant**

It is imperative to follow the operating instructions of the steam disinfectant. The treatment time is about 10 to 15 minutes.

**Never boil or autoclave the air tubing and the masks! We recommend to disinfect the masks with cold disinfectant.**

### **Drying**

Important for the hygiene at home is that all parts are thoroughly dried after cleaning. Let the individual parts air-dry or dry them with a lint-free cloth. For air-drying distribute the nebulizer parts on a clean and absorbent surface and do not cover them. Drying can be speeded up by using a hair-dryer.

### **Storage**

Between uses, and in particular during long breaks in treatment, wrap the nebulizer in a clean, lint-free cloth (e.g. a tea-towel) and keep it in a dry, dust-free environment.

### **Care of the compressed air tubing**

A disinfection of the compressed air tubing is generally not necessary. If you recognize any condensation, take the following steps:

Pull the compressed air tubing from the nebulizer and switch on your MicroDrop® compressor. Leave the compressor running until the air flowing through the tubing has removed any condensation.

In case of worse contamination or after one year at the latest you should replace the compressed air tubing for hygienic reasons.

**For hygienic reasons and in order to keep your nebulizer in good working order you should replace the following parts temporally limited. Yearly replacement of the nebulizer plus accessories e.g. compressed air tubing. For this purpose use our Year-Set.**

## **Hospital, doctor's practice and old people's home**

### **Principle**

The nebulizers are not sterilized before delivery and must be cleaned, disinfected and sterilized before first use. An effective cleaning / disinfection is imperative for an effective sterilization of the nebulizer. Please take care that within the bounds of your responsibility for the nebulizer's sterility during use only sufficiently validated procedures appropriate for your nebulizer are applied for the cleaning, disinfection and sterilization and that the validated parameters will be kept for each cycle. In addition, please follow the legislation of your country as well as the hygiene regulations of the hospital.

### **Cleaning / Disinfection**

Due to the clearly higher effectiveness, a mechanical procedure (disinfector) should be used if possible. Clean and disinfect immediately after use.

Treatment in ultrasonic bath is not recommended.

Any suitable cleansing agent / disinfectant can be used.

**Recommended method:  
mechanical procedure (disinfector)**

When selecting the disinfector, take care

- that the disinfector always has an effectiveness certificate (EN ISO 15883; CE-labeling),
- that – if possible – a certified “BGA” program **(at least 10 minutes at 93°C)** is used for the thermal disinfection (chemical disinfection can result in disinfectant residues in the nebulizer),
- that the selected program is suitable for the nebulizer and works with sufficient rinsing cycles,
- that rinsing is only done with sterile and low-germ water (maximum 10 germs / ml). Moreover, ensure a sufficiently low endotoxin and particle load (e.g. aqua purificata as per the regulations of the Pharm. Eur. Or USP).
- that only filtered air is used for drying
- that the disinfector is regularly maintained and verified.

When selecting the used cleaning agent system you have to take care

- that this system is basically suitable for the cleaning of the nebulizers (plastics),
- that – if no thermal disinfection is used – a suitable disinfectant with certified effectiveness (e.g. DGHM-VAH listing resp. CE-labeling) is added and that it is compatible with the used cleansing agent,
- that the used chemicals are compatible with the nebulizers.

It is imperative to follow the dosage recommendations from the manufacturer of the cleansing agent and – if necessary – of the disinfectant.

## **Procedure:**

1. Place the dismantled nebulizers into the disinfectant basket:  
A fixing of the nebulizers is only allowed, if they are still totally movable in the mounting; the nebulizers must not touch each other.
2. Place the disinfectant basket with the nebulizers into the disinfectant (Attention: do not stack several disinfectant baskets one on top of the other).
3. Start the program.
4. After the program has stopped, remove the disinfectant basket with the nebulizers from the disinfectant.
5. Assemble the nebulizers immediately upon removal and pack them.

## **Recommended method: manual procedure Cleaning**

1. Place the dismantled nebulizers into the cleaning solution for at least the time stated by the manufacturer of the cleansing agent / disinfectants.
2. Contamination on the outside can be carefully removed with a soft brush.
3. Intensively rinse the nebulizers at least five times in fresh, distilled or de-ionized water each; repeat this cleaning procedure, if the last rinsing solution was not clear or if there was still visible contamination on the product.

## **Disinfection**

1. Place the dismantled nebulizers in the disinfection solution for at least the time stated by the manufacturer of the disinfectant.

2. Intensively rinse the nebulizers at least 10 times in fresh, distilled or de-ionized water each; repeat this cleaning procedure, if the last rinsing solution was not clear or if there was still visual contaminations on the product.
3. Dry the nebulizers with filtered compressed air.
4. If possible, immediately re-assemble the nebulizers and pack them (see chapter "Storage").

**Possible alternative:  
chemical-thermal disinfection**

Place the individual parts of the nebulizer in the instrument disinfectant. Add a suitable cleansing agent / disinfectant and select the 60°C program.

**Storage**

Keep the sterilized nebulizer in a clean, lint-free place protected from contamination.

**Care of the compressed air tubing**

Generally, a disinfection of the compressed air tubing is not required. In the case of visible condensation in the tubing, take the following steps:

Pull the compressed air tubing from the nebulizer. Switch on the MicroDrop® compressor. Leave the compressor running until any condensation in the tubing has been removed by the air-flow.

In case of worse contamination or at least after one year the compressed air tubing has to be replaced for hygienic reasons.

**For hygienic reasons and in order to keep your nebulizer in good working order you should replace the following parts temporally limited.**

**Yearly replacement of the nebulizer plus accessories e.g. compressed air tubing.**

**For this purpose use our Year-Set.**

**It is absolutely important to carefully follow the instructions for cleaning and disinfection, in order to guarantee an impeccable function of the product and a successful therapy.**

## **Material resistance**

The nebulizer must not come into touch neither with aromatic or halogenized hydrocarbons and ethers nor – if possible – with aldehydes, esters, ketones, aliphatic hydrocarbons and oxidizing acids. The manufacturer cannot be liable for such cases.

Do not use temperatures above 121°C.

Repeated sterilization does not have any influence on the nebulizers, due to the properties of the material used; however the manufacturer accepts no responsibility, if the nebulizers are sterilized more than 50 times.

All parts of the nebulizers must be handled with care (prevention of mechanical damages).

## Packaging and sterilization

Only cleaned and disinfected nebulizers are allowed to be sterilized. If possible, the nebulizers should be re-assembled prior to sterilization.

Prior to sterilization the nebulizers must be packed in disposable sterilization packagings, which correspond to EN ISO 11607.

Alternatively, sterilization containers can be used. Prior to sterilization the nebulizers must be placed in a suitable sterilization container:

- as per DIN EN 868 / ANSO AAMI ISO 11607
- which are suitable for steam sterilization (temperature resistant up to 121°C, sufficient steam permeability) and regularly maintained.

### **Recommended method:**

#### **Steam sterilization**

- fractionated vacuum procedure (with sufficient product drying)
- steam sterilizer as per EN 13060 / EN 285
- validated as per EN ISO 17665 (valid commissioning and product specific efficiency evaluation)
- sterilization time at least 20 minutes at 121°C

It is absolutely not allowed to use the sterilization procedure with hot air (destruction of the nebulizer).

## Technical Data

Particle size: ..... MMAD 1.9  $\mu\text{m}$  (NaCl 0.9%) -API AEROSIZER.  
..... MMAD 4.9  $\mu\text{m}$  - INSITEC/MALVERN when  
..... using a MicroDrop® compressor  
Nebulization rate: ..... 0.28 ml / min. (NaCl)  
Filling capacity: ..... max. 6.0 ml  
Temperature  
resistant up to: ..... 121°C  
Guarantee: ..... 6 months starting from  
..... the purchase date

**CE** 0123 ..... This device corresponds to the  
..... regulations of the EC directives  
..... 93 / 42 / EEC for medical devices.

Manufacturer: ..... Respironics New Jersey, Inc.  
..... 5 Wood Hollow Road  
..... Parsippany, NJ 0705

Technical modifications reserved!

**Only use original spare parts for the  
*Jet* nebulizer!**

The nebulizer is suitable for use with all solutions and suspensions permitted for inhalation.

**Nebulized solution**

Budesonid

Albuterol

Beclametason Dipropionat

Tiopronin

Acetilcistein

Tiamfenicol Glicinat Acetilcistein

Flunisolid

Salbutamol

Ambroxol Clorid

## Order information

Description	REF
MicroDrop® <i>Family</i> Standard inhalation system for the whole family, complete with compressor, <i>Jet</i> nebulizer, adult mask, child mask, tubing, carrying bag and instruction manual	M 51603-00
Year-Set <i>Jet</i> nebulizer consisting of: <i>Jet</i> nebulizer, mouthpiece, filter, tubing, manual	M 53600-03
Nebulizer-Set <i>Jet</i> consisting of: <i>Jet</i> nebulizer with valve system, mouth piece, adult mask, child mask, tubing and instruction manual	M 53600-02
<i>Jet</i> nebulizer without tubing	M 53600-01
Tubing <i>Jet</i> nebulizer	M 50020-17900
Mask adult <i>Jet</i> nebulizer	M 40010-00400
Mask child <i>Jet</i> nebulizer	M 40010-00700
Nose piece <i>Jet</i> nebulizer	M 40010-01400
T-piece for connection of the <i>Jet</i> nebulizer to acapella. For combining inhalation therapy and physical vibration massage	M 50020-50600

## Order information

Description	REF
Compressor MicroDrop® <i>Family</i> incl. mains cable	M 50010-30300
Filter for compressor MicroDrop® <i>Family</i> 5 pieces	M 50010-30400
Carrying bag MicroDrop® <i>Family</i>	M 50020-34800
Instruction manual Jet nebulizer German / English	M 50010-42100
Instruction manual MicroDrop® <i>Family</i> Compressor German / English	M 50020-30500

## Warranty

This device comes with a warranty of 6 months from the date of purchase. The warranty period begins on receipt of the device. Any sales document, i.e. warranty card, sales slip or invoice, will serve as proof of purchase. Warranty claims must be made within the warranty period.

This warranty covers the free of charge replacement or repair of any parts that have become defective as a result of material or manufacturing defects.

The following are excluded under this warranty:

- damage from wear-and-tear or incorrect handling of the device (e.g. as a result of non-observance of the instructions for use)
- damage that can be attributed to maintenance or tampering by the purchaser or unauthorised third parties. The manufacturer shall not be held responsible for any direct or indirect damage caused by this device.

Services rendered under this warranty will neither lead to an extension nor a recommencement of the warranty period.

**In case of a warranty claim, please return the device or defective part, along with the stamped warranty card, to the following address:**

**MPV MEDICAL GmbH  
Wernher-von-Braun-Str. 1  
D-85640 Putzbrunn  
Germany**

**For any inquires, please call our  
Service Hot Line 0049 (0)89 46 17 23 71**

Device type: .....

Article no.: .....

Serial no.: .....

**Address of sender**

Name: .....

Street: .....

City / Town: .....

Phone: .....

Date of purchase: .....

from specialized dealer: .....

.....

**Description of defect**

.....

.....

.....

.....

.....

.....

.....

## Plan for nebulizer changing

It is important to change the nebulizer once a year for hygienic and therapeutic reasons.

Date of purchase	Date of purchase for a new nebulizer (at least once a year)	Sign of your Dealer

Please ask your dealer to help you with changing the nebulizer.

# MicroDrop® Family Vernebler MicroDrop® Family Nebulizer

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**Respironics New Jersey, Inc.**  
5 Wood Hollow Road  
Parsippany, NJ 07054

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