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PREFACE

Dear customer,

Thank you for the confidence you have shown in BURMEIER.

The cot WICKIE is factory-checked for electrical safety and function and has left our factory in faultless condition.

IMPORTANT – KEEP IN A SAFE PLACE FOR FUTURE USE – READ CAREFULLY!

This applies especially to the assembly phase but also to daily use.

It is also a practical reference guide. Keep the instructions in a handy place.

These instructions provide information for the operator and user regarding the convenient handling and safe operation.

We wish you and your users much success in care-giving and we are convinced that our product will make a positive contribution.

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GENERAL NOTES

The cot WICKIE is available in various models. These instructions may describe functions and accessories which are not included with your model.

Overview WICKIE Models:

Model Number: 58.1820.10

Wooden Reclining surface, 4 parts

Wooden side rails, 4 units, lowerable

Before putting the cot into operation for the first time:

■ Read these instructions carefully in order to avoid damage or incorrect operation.

■ Clean and disinfect the cot before first use.

According to the "Medizinprodukte-Betreiberverordnung (MPBetreibV) § 2, before using the cot, the user shall ensure that it is in proper working order and free of defects, and be aware of the instruction manual. This applies also to accessories.

The cot WICKIE complies with all requirements of the guidelines 93/42/EWG for medical products. It is classified as a dynamic medical product of the class I in accordance with "Medizinproduktegesetz (German abbreviation: MPG § 13, Medical Products Act)."

The cot WICKIE is checked by an independent examining institute. As with all technical, electrical devices improper handling can lead to damage and/or injury.

Observe your obligations as operator in accordance with medical devices "Medizinprodukte-Betreiberverordnung (German abbreviation: MPBetreibV, Operators of Medical Products Ordinance)" in order to ensure a dependable and safe operation of this medical device without endangering patients, users and third parties.

This manual contains safety guidelines, which must be observed. All persons who work with the cot WICKIE must familiarize themselves with these instructions and follow the safety guidelines.

DEFINITIONS OF INVOLVED PERSONS

In this manual the following persons are involved:

Operator

Operators (e.g.: Medical Supply Houses, Dealers and representatives, health insurance companies) are natural persons or legal entities, who utilize the cot WICKIE or authorize its use. The operator is responsible for instructing the user in the proper use of the unit.

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User

Users are persons who as a result of training and experience are authorized to assemble, perform maintenance and/or operate the cot WICKIE. Furthermore, the user can recognize and avoid possible dangers, and judge the clinical condition of the patient.

Patient

In this manual a patient is an infirm or handicapped person who reclines on the cot.

Specialist staff

Specialist staff are employees of the operator who are authorized as a result of their education and training to deliver, assemble, dismantle and transport the cot. Furthermore, they are familiar with the regulations for cleaning and disinfecting the unit.

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SAFETY INSTRUCTIONS

The cot WICKIE represents the latest technological development at the time of delivery and is checked by an independent examining institute.

■ Do not use the cot WICKIE if it is damaged.

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MEANING OF THE SAFETY SYMBOLS

In these instructions the following safety symbols are used:

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Warning about injuries to persons

Dangerous voltage. Life threatening.

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General danger. Injury or life hazard.

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Warning about property damages

Possibility of damage to drive, material or environment.

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Other symbols

Useful tip. For easier operation or better understanding of the unit.

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The safety symbol used does not replace the text of the safety note. Therefore read the safety instructions and follow them exactly!

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SAFETY INSTRUCTIONS FOR OPERATORS

- Remember your responsibilities in accordance with the MPBetreibV in order to ensure the safe operation of this unit without endangering patients, users and third parties.
 - Before first use, instruct every user in the safe operation of the cot in accordance with these safety instructions, which must be supplied along with the unit.
 - According to MPBetreibV 9 the user shall be notified of the location of this instruction manual.
 - Call attention to the dangers of improper use of the unit, especially in regards to the electrical drives and side rails.
 - Show the user how to use the locking mechanism for the hand switch and hand over the corresponding key.
 - The unit should only be operated by properly trained personnel.
 - Make sure that representatives are also familiar with the operation of the cot.
 - During long term use it is recommended that the cot be inspected for function and visible damage at regular intervals (Recommendation: yearly) (see chapter 6).
 - Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
 - When attaching other equipment (e.g. compressors for positioning systems) make sure that they are mounted and can function in a secure and safe manner.
 - Do not put multiple outlet electrical extensions under the cot. Leaking liquids can cause a fire hazard.
 - Pay special attention to:
Securely locating of all wiring, cables, tubes, etc.
 - ▶ See page 10 : *Intended use*
 - ▶ See page 21 : *Safety note for the electric drive system*
- For further information please contact the manufacturer of the accessories or BURMEIER directly.
- Make sure that your personnel follow the safety instructions.

SAFETY INSTRUCTIONS FOR USERS

- Let the operator instruct you in the safe use of the cot.

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- Before each use make sure that the cot is in proper and faultless condition.
- Take care that no obstacles, such as furniture or slanted ceilings interfere with the adjustment functions.
- Pay attention that when using additional electrical components, such as patient lifts, reading lights or compressors for positioning systems that their electrical cords do not get entangled or damaged by the moving parts of the bed.
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
- Make sure that when attaching other appliances (e.g. compressors for positioning systems) that a safe mounting and function is guaranteed.
- Do not put multiple electrical outlets under the cot. Leaking liquids can be a fire hazard.
- Pay special attention to securely locating all wiring, cables, tubes, etc.

For further information please contact the manufacturer of the accessories or BURMEIER directly.

- Take the cot out of operation if damage or a malfunction is suspected:
 - ▶ Immediately unplug the electrical plug from the wall outlet;
 - ▶ Clearly mark the cot as "DEFECTIVE";
 - ▶ Promptly notify the appropriate operator.

A checklist for assessing the proper condition is located in Chapter 6.

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- Check to make sure that the cot is not near to an open fire or strong heat sources, such as electric radiators or gas ovens.
- Do not continue to use the cot if any of the parts are broken, torn, damaged or missing. Only use replacement parts that are recommended by the manufacturer.
- Objects that could serve as a foothold or could represent a suffocation or strangulation hazard, such as strings, curtain cords, etc., should never be left in the cot.
- Never place more than one mattress in the cot.
- Position the cables of power adapter in such a way that during normal operation of the cot the cable will not be stretched, driven over or interfere with moving parts of the bed.

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■ Before any relocation of the cot it is imperative that the power cable is pulled from the wall socket and that the transformer unit is attached to the bed in such a manner that the cable cannot fall or be dragged over the floor.

■ Check the power cord regularly for damage (abrasions, exposed wires, kinks, pressure marks, etc.) in particular:

▶ after every larger mechanical strain (e.g.: rolling over the power cord with the cot or with an equipment cart; after a strong pulling or bending load caused by the bed rolling away with the power cord still attached to the wall outlet.)

▶ after relocating before plugging in the power cable.

▶ during prolonged operating by the user.

■ Do not use the area under the cot as a "parking spot" for any utensils.

■ The highest reclining surface position is provided to make the carer's work less strenuous and is to be used only during patient care procedures. The patient should never be left unattended with the reclining surface in this position, because the protective height of the side rail is inadequate.

■ Lower the reclining surface to its lowest position when leaving the patient unattended. This reduces the risk of injury to the patient getting in and out of the cot.

■ Keep the hand switch safe from accidentally falling during non-use (Hang it on the hook). Take care that the cable is not damaged by the moving parts of the cot.

■ To protect the patient and especially children from unintentionally adjusting the hand switch, place it beyond their reach (e.g. at the foot end) or lock the hand switch when:

▶ the patient is not able to control the bed safely or is unable to get out of dangerous positions without help;

▶ the patient could be endangered, because of unintentional adjustments to the drive motor;

▶ the side rails are up:
(There is a possible danger that limbs could be squeezed when adjusting the back and thigh rest);

▶ **unsupervised children are in the room with the cot.**

■ Adjustment functions can only be performed by a trained person!

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PRODUCT DESCRIPTION

NORMAL OPERATING PROCEDURE

■ The cot WICKIE, hereafter called bed, serves as a comfortable solution for positioning and easing the care of infirm, frail people in nursing homes. Additionally it was developed as a comfortable solution for in-home care of infirm, handicapped or frail people. It will support this care.

■ The WICKIE cot is not designed for use in hospitals.

■ This bed can be used for care under the instructions of a physician and can serve in diagnosing, treatment and observation of a patient. It is therefore equipped with a locking mechanism for the electrical adjustment functions.

■ This bed has no special connector possibilities for a potential equalization. Consider this before combining with additional electrically operated (medical) devices.

Further references about additionally applicable precautions can be found:

▶ In the instructions for the additional electrically operated devices (e.g. compressors for positioning systems, infusion pumps, nutrition probe ...)

▶ In the Standard DIN EN 60601-1-1:2002 (Safety of medical electrical systems)

▶ In the Standard VDE 0107:1994 (High current systems in hospitals)

■ The safe working load amounts to 1750 N (~ 175 kg), whereas a patient weight of 135 kg has been allowed. The remaining 40 kg are allocated for the mattress and attached accessories.

■ Due to the design and construction of the cot, the ground clearance/foot room beneath the bed has been reduced to 12 cm in some places.

■ Observe, especially with patients in poor health condition, the safety tips in chapter 4.

■ This bed may only be operated by trained personnel.

■ This bed is useful for multiple re-use. Bear in mind the necessary requirements:

▶ See chapter 5;

▶ See chapter 6.

■ While the patient also is lying in bed, it is allowed to drive the bed within the building.

This bed may only be operated under the operating instructions laid out in this instruction manual.

Any other use is deemed to be inappropriate.

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Sale of this product in North America, especially in the United States of America, is prohibited. Distribution and use of the cot, including distribution and use by third parties in these markets is forbidden by the Manufacturer.

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STRUCTURAL DESIGN

The bed is delivered in parts for self-assembly. This means that it can also be delivered without difficulty to flats in apartment buildings. The bed consists of a peripheral wooden frame, which stands on four steerable, individually locking castors and which supports the side rails.

The reclining surface rests on a height-adjustable base frame. The reclining surface and base frame are integrated in the wooden surround.

Reclining surface frame

The reclining surface frame has four sections: a movable backrest, a firm middle section, a moveable thigh rest and lower leg rest. The back- and thigh rest can be adjusted with the aid of electric motors. The horizontal height of the reclining surface can be adjusted and inclined to a low foot end position. All adjustment functions are controlled by a hand switch.

Side Rails

The bed has side rails on all sides which can be raised as a barrier or lowered when not needed. This safeguards the patient from accidentally falling out of the bed.

Electric adjustment system

The electrical adjustment system of this bed is error protected, flame-retardant (V0) and consists of:

- a power adapter.

The power adapter consists of wall plug and a low voltage connection cable.

The power adapter requires 230 V and creates a 24 volt low voltage, which is safe for patient and user.

The power adapter supply all drive motors with the 24 volt safety low voltage through cables and a wiring harness. The connecting socket on the base frame is moisture proof.

- a hand switch with a sturdy hook.

The user can lock the adjustment functions of the hand switch if the condition of the patient warrants it.

- one drive motor with integrated control for the horizontal height .
- one drive motor with integrated control for the thigh rest.
- one drive motor with integrated control for the backrest.

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DESCRIPTION OF MATERIALS

The bed is mostly constructed out of steel sections, their surfaces have been polyester-powdercoated, or with a metallic coating of zinc and chrome. The end boards, the side rail beams and the spring wood slats consist of wood and derived timber products whose surface has been sealed.

All surfaces are harmless to skin contact.

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ASSEMBLY AND START UP PROCEDURE

The cot is shipped disassembled.

The assembly is taken care of at the destination by qualified personnel supplied by the operator.

Assembly can be done by one or two persons.

The cot ships in seven cartons:

Carton 1 / 2

Head and foot sections with cover caps and fixing materials

Carton 3

Side boards

Carton 4

Base frame with motor, wiring harness, hand switch, cross braces, power adapter

Carton 5

Reclining surface, including motors, instruction manual

Carton 6

Inner frame, fixing materials

Carton 7 / 8

Side rails for head and foot sections

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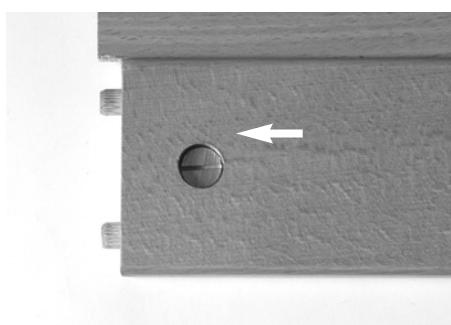
ASSEMBLY

ASSEMBLY OF THE WOODEN SURROUND WITHOUT SIDE RAILS

☞ Lay the side sections down with the drilled holes facing upwards.

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

Take care during assembly that

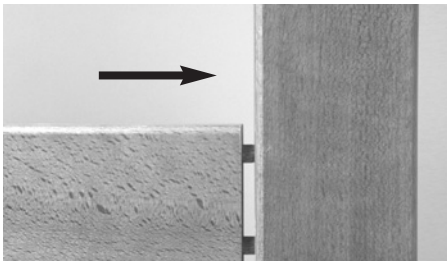- the drilled holes with the nuts face inwards,
- the pre-drilled holes for the support bars are at the bottom.

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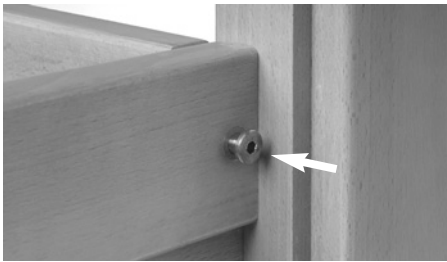
☞ Fit the head section onto the side sections.

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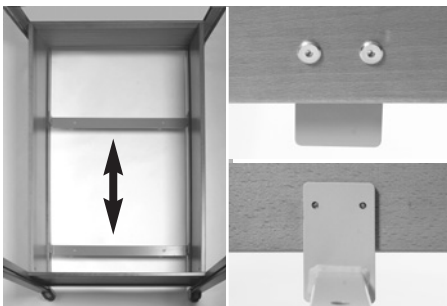☞ Fix the side sections to the head section using the threaded rods and sleeve-nuts.
☞ Fix the side sections to the foot section using the threaded rods and sleeve-nuts (5 mm hexagon socket screws).

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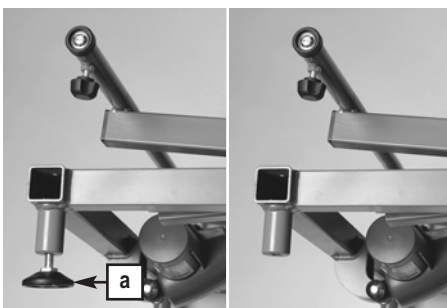
☞ From the outside, screw the two support bars to the side sections (4 mm hexagon socket screws).

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FIXING THE BASE FRAME IN THE WOODEN SURROUND

☞ Unscrew and remove all the feet (a) from the support structure of the base frame.
☞ Carefully remove the protective sleeves from the spiral cables.

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Observe without fail:

Ensure that the insulation of the spiral cables is not damaged.

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☞ Set the base frame into the wooden frame.

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
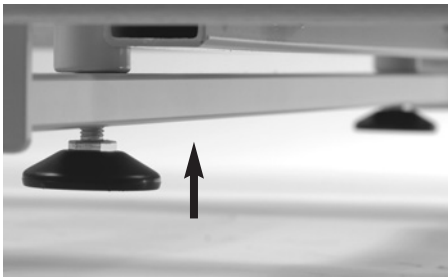
Notice:

Note that the support bars are positioned asymmetrically. The side with the lifting motor must be placed on the support bar that is closest to the foot section of the wooden frame.

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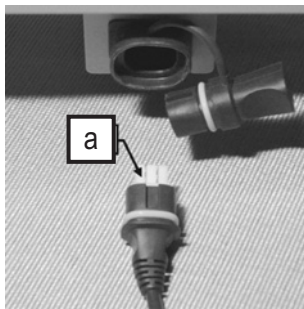☞ Fix the base frame firmly to the support bars using the previously removed screw feet.

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☞ Take the transformer unit out of its wrappings.

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☞ Pull the plug out of the connecting socket.

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☞ Plug the 6-poled plug (a) of the transformer unit in this connecting socket. The 6-poled plug has a safety groove. Press the plug firmly into the connecting socket.

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☞ Plug the power adapter of the transformer into the wall socket.

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☞ On the hand switch, push on the left button of the middle pair.

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The scissors of the base frame now drive upwards as long as you press the button.

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☞ Drive the lifting frame to its highest position.

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The two frame connectors are secured to the top end reclining surface frame before leaving the factory.

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Please note, that the frame connectors for the right and left side are different.

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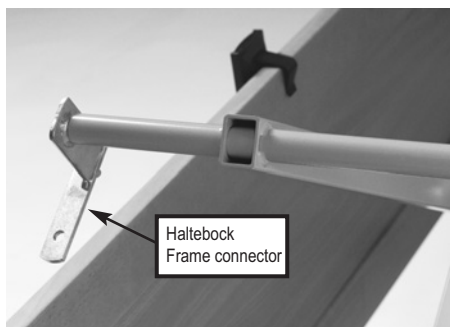
☞ Remove one frame connector from the reclining surface frame by loosening the knurled screws.

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Leave the other frame connector secured in position!

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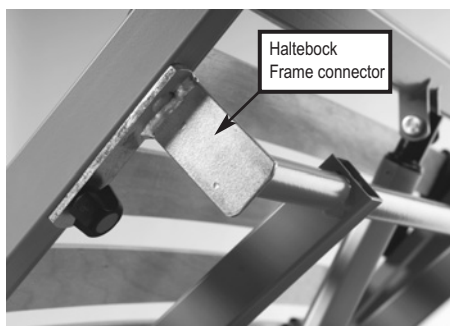
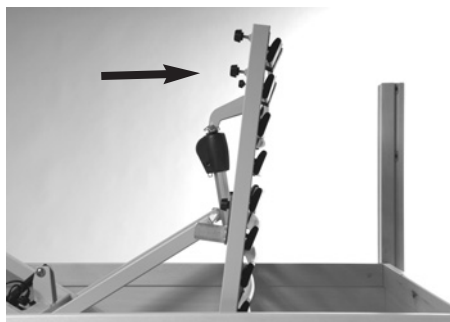
FIXING THE RECLINING SURFACE TO THE BASE FRAME

☞ Insert the support tube of the removed frame connector into the open end of the cross tube.

The inside of the cross-tube is lubricated.

If you accidentally pull out the frame connector, do not wipe off the lubricant!

☞ Take the head part of the reclining surface frame and push the support tube of the frame connector into the other open end of the cross tube.

☞ Replace the opposite frame connector and secure to frame.

The pin of the frame connector must be located in the hole of the reclining surface frame. Tighten with the knurled screw.

☞ Swivel the head part of the reclining surface frame backwards through 90°.

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
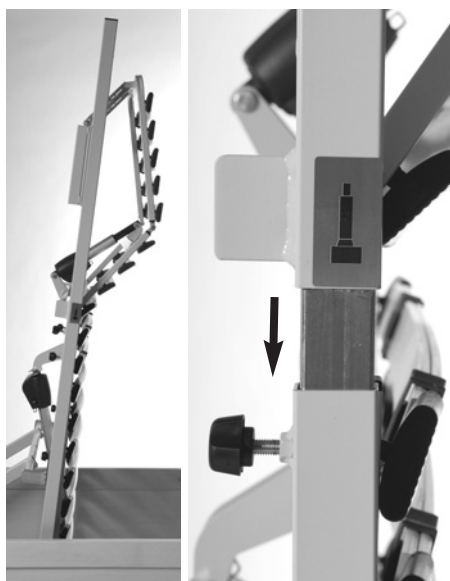
Observe without fail:

When the foot part of the reclining surface is lifted into a vertical position, the thigh rest could fold down. In order to avoid injuries, the thigh rest must be folded out before fitting the foot part to the head part.

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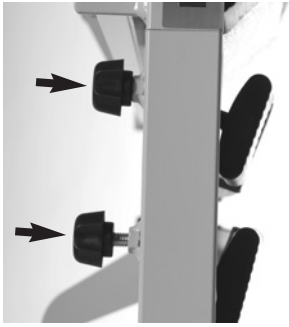
☞ Fit the foot part of the reclining surface frame to the head part.

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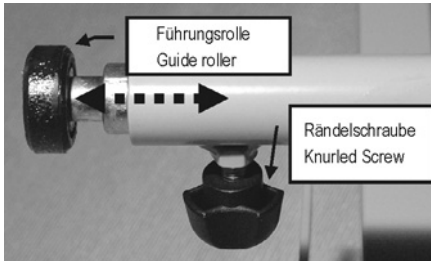☞ Fix the head and foot parts at both sides with the screws.

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☞ Loosen both knurled screws of the guide rollers. Do not unscrew the knurled screws completely, but leave them in the threads for about 2 revolutions.

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☞ Completely insert the two guide rollers into the tube to the limit and leave them there.

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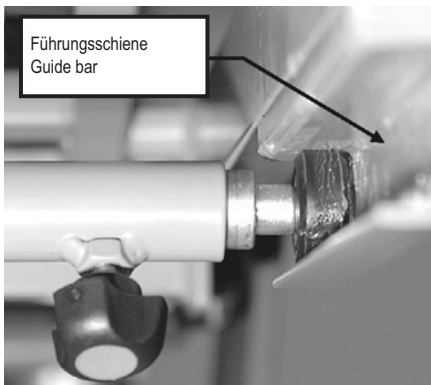
☞ Fold the complete reclining surface frame into the horizontal position.

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☞ Take the weight of the reclining surface frame and withdraw the guide rollers in turn until the mark appears. Align the guide rollers into the left and right guide bar.

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☞ Tighten the 2 knurled screws **securely** ! Do not use pliers to tighten the screws !

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Take special note:

Check the correct position of the guide rollers in the two guide bars. Ensure that both guide rollers are running freely in the guide bars. If necessary adjust sideways equally.

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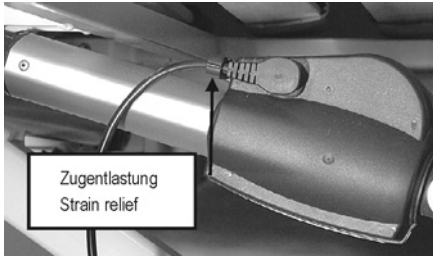
☞ Connect the drive unit of the foot part to the wiring harness.

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☞ Connect the drive unit of the head part to the wiring harness.

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Secure the cable by clicking the strain relief grommets firmly into the corresponding openings.

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☞ Now plug the power adapter of the transformer into the wall socket.

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Observe without fail:

When laying the cable in the base frame, make sure that through the up and down movement no shearing or squeezing damages the cables.

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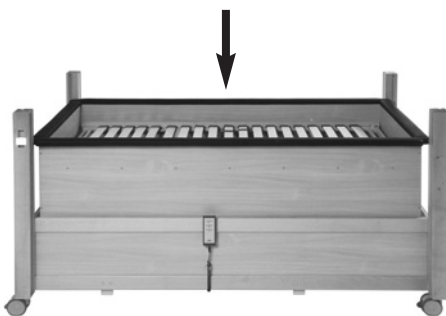
FITTING THE INNER CLADDING TO THE RECLINING SURFACE

☞ Push the wooden dowels into the corresponding drilled holes in the long inner side-cladding sections.

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☞ Place the inner cladding onto the reclining surface from above.

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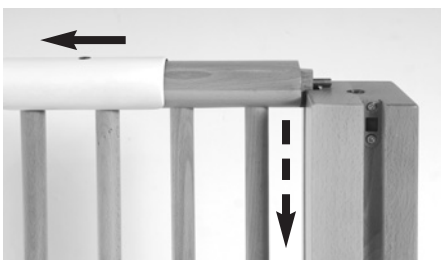
ATTACHING FIXED SIDE RAILS

☞ Take the first fixed side rail and fit it into the guide slots in the head and foot sections, from above.

☞ Slide the locking grips to unlock the side rail.

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
☞ Mount further fixed side rails in the same way.

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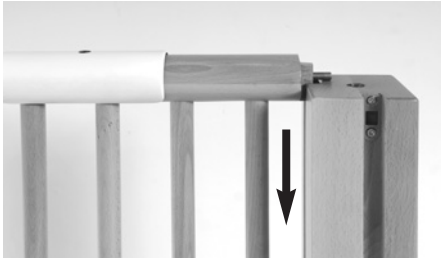
- ☞ Take the first telescopic side rail and fit it into the guide slots in the bed frame, from above.
- ☞ Slide the locking grips to unlock the side rail.
- ☞ Slide the side rail in completely.

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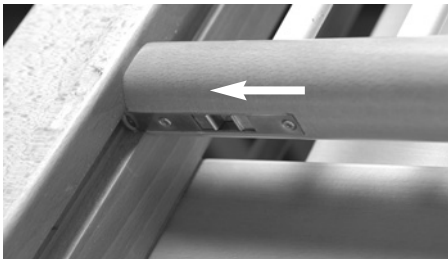
- ☞ Slide the telescopic side rails down to the lowest position.

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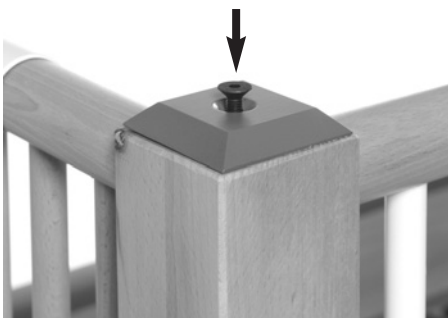- ☞ Slide the bottom bolt into the slot.
- ☞ Make sure that the bottom part of the telescopic side rail cannot be raised beyond the final stop position.

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- ☞ Screw the end caps onto each of the four bed posts.
- ☞ Then push the cover caps onto the screw heads.

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LOCATION REQUIREMENTS

■ There must be enough space available to be able to adjust the bed. No furniture, window sills and the like should be in the way.

■ The space under the bed must remain clear.

■ Check before use on parquet flooring, that the castors do not cause discoloration of the varnish. Use on ceramic tiles, carpet, laminate and linoleum is harmless.

BURMEIER is not liable for any damage to floors due to daily operation.

■ A properly installed 230 volt power outlet must be available, near the bed.

■ Take care when attaching further accessory equipment (e.g. compressors for positioning systems, etc.) that their attachment is safe and their functionality is not impaired. Especially important is hereby also the safe placement of all movable connecting cables, tubes, etc. If you have any further questions or need clarification contact the manufacturer of the accessory equipment or BURMEIER.

Observe the following points when setting up the bed to minimize possible fire hazard. Instruct the user and the authorized personnel in these points!

■ If possible use only fire-retardant mattresses and bed sheets.

■ Only use appropriate firm mattresses according to DIN 13014. This mattress should be inflammable according to DIN 597 Section 1 and 2.

■ Avoid smoking in bed because of the danger of fire.

■ Use only technically faultless appliances (e.g. heating blankets) and other electronic devices (e.g. lamps, radios).

■ Take care that these devices are only used for the purpose intended and that they cannot accidentally get on top of or underneath the bed sheets (Fire hazard because of overheating).

■ Absolutely avoid the use of extension cords or multi outlet extension cords under the bed (Leaking liquids can cause a fire hazard).

■ Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.

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REACHING OPERATIONAL READINESS

- After successful assembly, complete an inspection according to chapter 6.
- Clean and disinfect the bed before first time use and before every further use according to chapter 5.

After successfully completing and heeding all steps on page 12 through 19 the bed is now operationally ready.

DISASSEMBLY

- ☞ Drive the bed to its highest horizontal position.
- ☞ Lock all four castors using the brakes.
- ☞ Remove all four end caps from the bed posts.
- ☞ Unlock the telescopic side rails.
- ☞ Remove all the side rails.
- ☞ Remove the inner cladding that rests on the reclining surface.
- ☞ Pull out the tension relief grommets and plugs from the drive motors of the head and foot sections.
- ☞ Loosen the knurled screws on the reclining surface frame at the foot end.
- ☞ Push the guide rollers into the tube so that the reclining surface frame lies freely on the base frame.
- ☞ Unscrew one frame connector from the reclining surface frame.
- ☞ Remove the reclining surface frame from the base frame.
- ☞ Drive the bed to its lowest horizontal position.
- ☞ Pull the mains plug of the power adapter out of the socket.
- ☞ Pull the power cable plug out of the connecting socket.
- ☞ Remove the feet of the base frame from the support bars by unscrewing them.
- ☞ Lift the base frame out of the wooden frame.
- ☞ Dismantle the wooden surround.

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SPECIAL SAFETY INSTRUCTIONS FOR THE DRIVE UNIT

- Take care when adjusting the bed that no limbs of patients, users or any other persons, especially children, can be jammed or injured by getting under the movable parts or the bed frame.
- When the side rail beams are in the raised position, always lock the hand switch to ensure the safety of the patient by preventing unwanted motorised adjustments to the height and to the back and thigh rests. (Danger of getting limbs trapped when adjusting the back- and thigh rests).
- When using accessories with the bed it is necessary to: make sure that the arrangement of these accessories will not cause any places that might squeeze or hurt the patient. If this is not the case the operator must safely disable the adjustment function of the back- and thigh rests.
 - ▶ To do so place the hand switch out of the patient's reach (e.g. foot section) or lock the adjustment functions on the hand switch.
- Make sure that the power and hand switch cable cannot be jammed or damaged in other ways.
- Make sure that before relocating the bed, the power cable cannot be stretched, rolled over or damaged in any other way.

- With every relocation, pull the power cable out off the wall socket and position the power adapter in a safe place so it cannot fall.
- Make sure that the connecting socket is always closed with the stopper when it is disconnected from the power adapter.
- Make sure that when adjusting the bed no obstacles such as furniture or slanted ceilings are in the way. This will prevent damage.

- ◆ An electronic overload detector switches off the central control unit, when the load is too great. After clearing the overload and pressing the hand switch again the central control unit will work again.

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- ◆ A continuous run time of 2 minutes should not be exceeded. Afterwards a pause of no less than 18 minutes should be observed. Another possibility: 1 minute continuous run time and 9 minutes pause.
- ◆ If the maximum continuous run time is not heeded (e.g. by constantly "playing" with the hand switch) a thermal safety component switches off the power for safety reasons.
- ◆ The adjustable range of every function is electrically/mechanically possible only within a predetermined range.
- ◆ Electrical disturbances affecting other nearby electronic appliances (e.g. crackling on the radio) cannot be excluded, despite adherence to all prescribed regulations. In such rare circumstances increase the distance between the appliances, do not use the same power socket or switch the affected appliance off for a while.

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HAND SWITCH

The electrical adjustments can be made using the hand switch [8].

The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

Locking mechanisms have been built into the hand switch for safety reasons. For the safety of the patient, adjustment functions can be locked at the hand switch, if, from a clinical point of view and with the approval of the attending physician, the condition of the patient warrants it.

- ◆ The hand switch, with its elastic hook can be hooked onto the bed.
- ◆ The spiral cable allows for greater mobility.
- ◆ The hand switch is water-tight and washable.

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- ◆ Heben
- ◆ Senken

OPERATING THE HAND SWITCH

- Train the patient in the functions of the hand switch.
- The drive motors will run as long as the button is held down.
- The adjustment functions, except for the lowered foot position, can be used in both directions.

■ Basically the functions of the buttons are as shown :

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Adjusting the backrest

Push this button if you want to adjust the backrest angle.

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Adjusting the reclining surface height

Push this button if you want to adjust the height of the reclining surface.

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Adjusting the thigh rest

Push this button to adjust the thigh rest angle.

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Only the operator is authorized to use the locking function!

In case the clinical condition of the patient is so critical that any adjustment of the beds by means of hand switch causes danger, the operator has to disable the bed immediately. The cot remains in the position it had at the time of disconnecting.

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Do not turn the locking key with force over the stop of the lock! The lock or the whole hand switch can be damaged.

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The locking key is delivered attached to the bed with a cable tie on the hand switch. The locking key is not intended for use by the patient. It must be removed from the hand switch. The operator or a person authorized by the physician should keep the **locking key**.

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■ Turn the respective lock on the hand switch with the locking key clockwise in the locking position.

The color of the respective display will change from green to yellow.

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Lock is vertical

Color of the display is: **green**

Buttons can be pushed ("Clicking sound")

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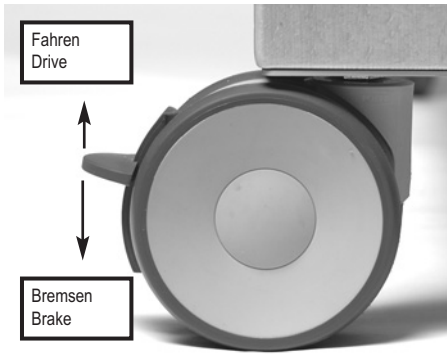
Drive disabled

Lock has been turned clockwise by about 15°

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CASTORS

The bed stands on four steerable, individually-locking castors [11].

- ☞ **Braking:** Push the foot lever down.
- ☞ **Driving:** Lift up the foot lever using your foot.

- Relocate the bed only when it is positioned in the lowest position.
- Before relocating the bed, ensure that the transformer unit is securely positioned on the bed so that it cannot fall off.

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LOWER LEG REST

When the thigh rest is fixed, the lower leg rest [5] can be individually adjusted. For this reason there are two adjusting ratchets, which will lock the lower leg rest into position.

Raising

☞ Raise the lower leg rest on the foot end - not with the mattress brackets - up to the desired level. The lower leg rest will automatically snap into place.

Lowering

- ☞ First lift the lower leg rest all the way to the stopping point.
- ☞ Slowly lower the lower leg rest.

There is a potential for injury if the lower leg rest falls straight down.

If you lower the thigh rest, the lower leg rest will be lowered automatically.

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SIDE RAILS

Side rails conveniently prevent the patient from falling out of bed accidentally. They do not, however, prevent a patient from leaving the bed.

Engaging / Disengaging

- ☞ Take hold of the locking grips and lift the side rail slightly.
- ☞ At the same time, slide the locking grips towards the middle.
- ☞ Slide the side rail to the desired height.
- ☞ Then release the locking grips and slide the side rail until it snaps into the next locked position.
- ☞ Check that the side rail beams are locked into place by pushing on them from above.

Special safety notes for use of side rail beams:

- Before you leave your child unsupervised in the cot, always check to make sure that the adjustable side rails are in a fixed position.
 - Only use safely lockable, technically perfect, undamaged side rail beams with proper gap dimensions
 - Make certain that only suitable Original BURMEIER side rail beams are used, which have been approved for use with the bed model you are using or that are already integrated in the bed.
 - Before using side rail beams evaluate and take into account the clinical condition and any special problems of the patient
 - ▶ For especially small, frail patients, additional protective measures for reducing the distance between bars on the side rails may be necessary. In these cases use, for example side rail foam covers (accessories, see chapter 8), retaining belts, etc.
- This will effectively ensure patient safety by reducing the risk of becoming trapped or slipping through.
- Only use appropriate, firm mattresses accord to DIN 13014 with a height of 10 to a maximum of 15 cm. Further, this mattress should be fire resistant according to DIN 597 Section 1 and 2.
 - If elevated patient positioning systems are employed, such as antidecubitus mattresses, an effective side rail height of at least 30 cm above the unweighted mattresses must be ensured. If this requirement is not met, the operator must take suitable measures to safeguard the patient. This can be done by using an attachable bar for side rails (Accessory).

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■ The mechanical parts on the bed frame and the side rails which serve to protect the patient must be regularly examined for damage. This should not only happen before the installation of the side rail but also during use in shorter intervals (at least before every new assignment).

■ When the side rails of electrical beds are in the raised position, always lock the hand switch to ensure the safety of the patient by preventing unwanted motorised adjustments to the height and to the back and thigh rests. Otherwise there is a risk of trapping limbs when the back and thigh rest are adjusted.

■ Before lowering the side rail beams make sure that the patient's limbs are clear of the side rail beams. This way you can avoid injuring the patient.

If these safety warnings are not observed there is a danger for the patient of being injured by the side rails, of falling out of bed because of too large a distance between the bars or because of damage, the use of unsuitable side rails or the incomplete locking of the side rail.

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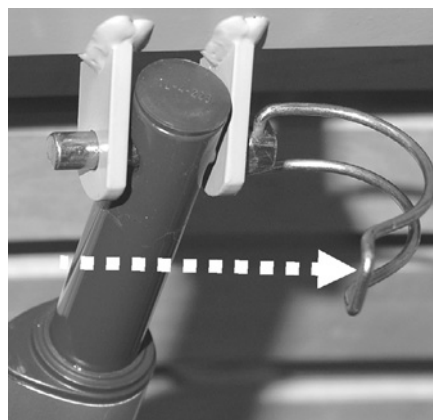
EMERGENCY LOWERING OF THE BACK REST

The upright back rest can be lowered manually in case of a power outage or failure of the power supply or the electrical drive systems.

In this case two users are absolutely necessary!

Disregarding these safety guidelines and instructions may cause the backrest to fall out of control, which could lead to serious injuries for the user and the patient!

Emergency lowering may only be carried out in the case of extreme emergency and by users who have a complete command of the procedures below.

We strongly advise you to practice emergency lowering of the backrest under normal conditions so that in case of an emergency you will be able to react quickly and properly.

☞ Before the emergency lowering, ease the load on the backrest.

☞ The first user raised the back rest slightly by the outside edge of the head section and holds it in this position.

☞ The second user now removes the spring pin from the lift rod. To do this swing away the bent bracket and pull the spring pin along with the bent bracket out of the lift rod of the back rest motor.

☞ The motor has now been detached from the back rest and swings down.

☞ Now the first user carefully lowers the back rest.

If the back rest falls out of control, the patient and the second user may be injured!

☞ To convert the bed back to its normal operating state: Swing up the connecting mount and secure the spring pin, then turn over the bent bracket.

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CLEANING AND DISINFECTING

GENERAL

Cleaning is the most important requirement for a successful chemical disinfection.

Generally a routine cleaning of the bed, if used by the same patient, is hygienically sufficient.

Disinfection of the bed frame is only necessary in case of visible contamination with infectious or potentially infectious material (blood, stool, pus) or by presence of an infectious disease under the direction of a physician.

The bed must be cleaned and washed down with disinfectant in case of a patient change!

Before beginning the cleaning, please note:

- Unplug the power cable from the wall socket and store it in such a way that it does not come into excessive contact with water or other cleaning solutions (store in plastic bag).
- Ensure that all plugs on the transformer unit and the drive motors are plugged in properly.
- None of the electrical components must show any visible damage; otherwise water or cleaning solution could penetrate. This could lead to malfunction or damage to the electrical components.
- Before operating the bed again, make sure that there is no residual moisture on the electrical contacts, by drying or blowing on the power plug.
- The electrical components must not be cleaned with a water jet, a high pressure cleaner or other similar device! Clean only with a moist towel.
- If you suspect that water or any other form of moisture has penetrated the electrical components, pull the power plug out of the wall socket immediately. If already disconnected from the power supply, make sure it is not plugged in again. Clearly mark the cot as "Defective" and take it immediately out of service. Report this to the operator promptly.
- Failure to follow these directions could lead to major damage of the bed and its electrical components and lead to subsequent malfunctions.

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CLEANING AND DISINFECTION PLAN

■ Remove the bed sheets and give them to the laundry.

■ Clean all surfaces including the slatted boards and the reclining surface made of plastic inserts or wire mesh base with a mild and environmentally safe cleaning solution. This also applies to the hand switch.

■ If the bed has visible contamination e.g. infectious or potentially infectious material has polluted it, it must be disinfected by wiping after cleaning. Use approved disinfectants for each surface according to the products listed by the DGHM (German Organization for Hygiene and Microbiology). This also applies to all beds from patients with illnesses subject to report according to §6 of the infection control law (German abbreviation: IfSG, Protection against Infection Act), colonization or infection with treatment resistant pathogens (e.g. MRSA, VRE) and all beds from the intensive care and infection control units. Use only the DGHM approved disinfectant in the approved concentration.

■ Disinfecting the castors is only necessary when visibly contaminated by infectious or potentially infectious material.

Notice: Continuous disinfection is only necessary for patients with a treatment resistant pathogen (e.g. MRSA) within the hospital.

INSTRUCTING THE USER AND QUALIFIED PERSONNEL

To guarantee the correct procedure for cleaning and disinfection is followed, we recommend to correctly instruct the handler and qualified personnel. Thereby communicating that the following points need to be observed:

■ The clean bed is to be transported to the patient's home in such a way that it does not become polluted or contaminated.

■ When disassembling the bed we recommend cleaning and disinfecting it right away. The qualified personnel should be familiar with the steps necessary to thoroughly clean and disinfect (the operator will provide the individual steps necessary). It is necessary to use only the DGHM approved disinfectant in the approved concentration. The disinfectant must be appropriate for the surface being cleaned.

■ The qualified personnel must be equipped with water-tight one-time-use aprons and gloves.

■ Only fresh and clean towels must be used to clean and disinfect, these must then be given to the laundry.

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- After cleaning the bed, the qualified personnel must also disinfect their hands before going to the next patient. An appropriate disinfectant dispenser should be part of the qualified personnel's gear.

The immediate cleaning of the bed while disassembling has the benefit of not transporting "dirty" beds or bed parts along with clean ones in the same truck. A transfer of potentially infectious spores which may be clinging to the used bed may be avoided. A transfer of spores of a hospital-acquired infection will be avoided with strict adherence to these recommendations.

If the beds are to be stored for a while, they should be covered to keep them free of dust, accidental pollution and contamination.

CLEANING SUPPLIES AND DISINFECTANTS

Make use of these recommendations to keep the bed serviceable for a long time.

- The surfaces must be intact; any damage should be repaired immediately.

- We recommend a (damp) wash down. When choosing the cleaning solution make sure it is mild (gentle to skin and surfaces) and environmentally friendly. In general a regular household cleaner can be used.

- For cleaning the coated metal parts use a wet cloth with regular household cleaners.

- The disinfectant used for the wash down should be one approved for the purpose and listed by the DGHM (in the recommended dosage). Disinfectants and concentration listed by the Robert-Koch-Institute (the former Bundesgesundheitsamtes) should only be used in case of an epidemic according to the infection safety law and only on the orders of the appropriate health department.

- Despite the excellent mechanical resistance of the bed, scratches, markings, etc. which penetrate the coating should be resealed using a suitable medium to prevent the penetration of moisture. Contact BURMEIER or your specialized dealer with any questions.

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■ Do not use scouring agents, stainless steel cleaners, abrasive cleaning agents or scouring pads. These substances can damage the surface.

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Notice:

Surface disinfectants based on aldehyde generally have the advantage of working on a large array of pathogens, have a relatively low protein error and are environmentally friendly. Its main disadvantage is the allergy-causing and irritation potential. Compounds on the basis of glucoprotamine do not have this disadvantage and are very effective, though mostly more expensive. Disinfectants on the basis of chlorine splitting combinations, with prolonged contact or in too high doses, can be corrosive to metals, plastics, rubber and other materials. In addition to this, the compounds have a high protein error, are irritants to the mucous membranes and are not environmentally friendly.

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CONTACT WITH DISINFECTANTS

- Follow the guidelines for each product and manufacturer. Observe the exact dosage! We recommend automatic dosage apparatus.
- Always mix the solution with cold water, to avoid vapors that aggravate the mucous membranes.
- Wear gloves to avoid direct skin contact.
- Do not store the mixed disinfectant in open containers with the cleaning rag inside. Close the container!
- Use closeable bottles with dosage pumps for wetting the cleaning rag.
- After finishing the surface disinfection ventilate the room.
- Use a wipe down disinfecting method, do not use a spray. Using the disinfectant as a spray causes the disinfectant to turn into a mist which could be inhaled.
- The wipe-down disinfection plays a major part.
- Do not use alcohol for larger areas.

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operators of cots are obligated to keep medical products in a safe state throughout their entire service life. This also includes regularly carrying out expert maintenance as well as safety checks.

Notice for Operators

This bed has been constructed and built to last safely for a long time. With appropriate operation and use, this bed has a life expectancy of 2 to 8 years. The life expectancy is based on the conditions and frequency of use.

Through repeated transport, assembly and disassembly, improper operation as well as long time use, it is possible that damage, defects and deterioration can occur. These deficiencies can lead to danger if they are not recognized and quickly fixed.

Because of this there are legal bases for the regular enforcement of tests to ensure the safe condition of this medical product.

According to § 4 of the Medizinprodukte-Betreiberverordnung operators are obliged to maintain the product. Because of this the following regular inspection and function controls are to be performed by the operator as well as the user.

Point out these necessary tests to the user.

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BY THE USER

Along with the regular substantial tests by technically qualified personnel, the normal user (nurses, supporting relatives, etc.) must also perform a sight and function check at shorter, regular intervals, as well as before a new patient.

- If damage or a malfunction is suspected, take the bed out of service immediately and disconnect it from the power until an exchange or repair can be made!

- Contact your operator regarding exchange or repair of faulty parts.

Recommendation:

All electrical and mechanical components should be tested monthly. The main power cable as well as the hand switch cable should be checked after every mechanical load and after every relocation.

For this purpose use the checklist on the following **page 35**.

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Checklist: Test protocol by the user

Test	OK	Not OK	Defect description
Visual check of electrical components			
Hand switch and cable	Damage, Cable Routing		
Power cord	Damage, Cable Routing		
Hand switch	Damage, Foil		
Transformer unit	Damage, Cable Routing		
Visual check of mechanical components			
Bed frame	Damage, Deformation		
Spring wood slats	Damage, Splinters		
Bed box	Damage, Splinters		
Reclining surface frame	Damage, Deformation		
Side rail	Damage, Splinters		
Function test of electrical components			
Hand switch	Functiontest, Blocking function		
Strain relief, Power cable of power adapter	Tight fit		
Function test of mechanical components			
Emergency lowering of backrest	Test according to instruction manual		
Castors	Free movement an locking		
Knurled screws	Tight fit		
Side rail	Safe locking/unlocking		
Lower leg rest	Latching		
Accessories (e.g. trapeze pole, trapeze)	Secure, Damage		
Signature of the tester:	Result of the test:		Date:

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BY THE OPERATOR

The operator of the cot according to MPBetreibV § 4 is obligated with every new assembly, every maintenance and throughout the operation, to perform regular inspections, to ensure the safe condition of the cot.

These tests are to be repeated during regular maintenance and depending on the use according to MPBetreibV § 4 and from the Directive of the German Employers Liability Insurance Association according to DIN EN 62353.

♦ Observe this sequence for the inspection according to DIN EN 62353:

I. Inspection

II. Electric Measurements

III. Function Test

♦ The performance of the function test as well as the rating and documentation of the test results can, according to MPBetreibV § 4 only be performed by experts with the appropriate knowledge, training and experience as well as the tools necessary to perform it according to the rules.

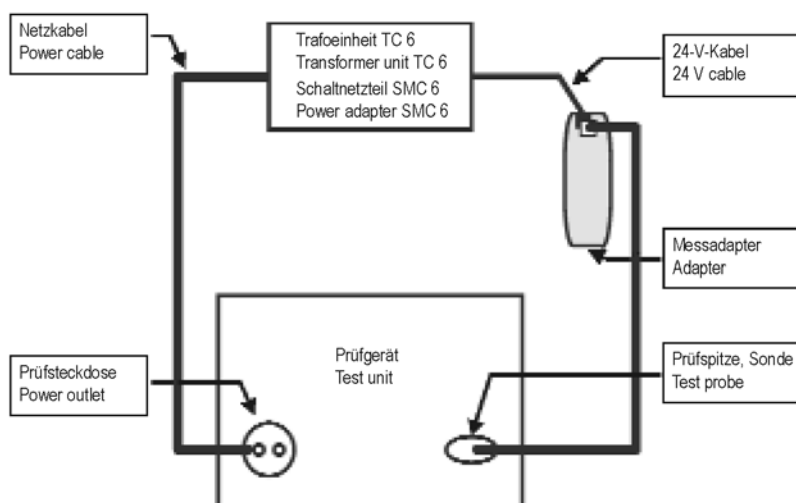
♦ The performance of electrical measurements according to DIN EN 62353 can, with availability of proper measuring equipment, also be performed by an electro-technically trained person [according to BGV A3] with medical and appliance specific training.

♦ The assessment and documentation of the test results can only be performed by an expert with the corresponding knowledge, training and experience.

♦ Testing of the power adapter requires only measurement of the output.

Thus, after replacing the transformer with a new unit, the bed can be put to use.

♦ BURMEIER offers power adapter testing as customer service. Send the power adapter back to BURMEIER and receive a tested replacement power adapter. Contact us for further details; Address see page 42.

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Preparation:

- Unplug the power plug of the power adapter from the wall socket.
- Unplug the 24 volt cable from the connecting socket.
- Plug the plug of the 24 volt cable in the adapter (extra accessory; request from BURMEIER).
- Connect the adapter with the test probe of the test unit.
- Plug the power plug of the power adapter into the power socket of the test unit.

Test method:

- Output power test: direct or differential according to DIN EN 62353
- Output power test according to the test unit manufacturers instructions.

Limit:

- Output power less than 0.1mA.

Test cycle:

We recommend a yearly visual inspection and performance check. If this checks was been ok, an electrical testing suffices every ten years.

The test cycle can maximally be extended to a total of two years, according to demonstrably observing the 2%-error-quota (see BGV A3: § 5, table 1 B).

If damage or a function error is suspected take the cot out of service immediately and disconnect it from the power until an ex- change or repair can be made!

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The test protocol draft on **pages 40 and 41** can be used.

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Customer / med. Facility / practice:						
Address:						
Carried out: <input type="checkbox"/> Repeat inspection		<input type="checkbox"/> Inspection prior initial operation				
<input type="checkbox"/> _____		<input type="checkbox"/> Inspection following repairs / servicing				
Equipment type: <input type="checkbox"/> Hospital bed <input checked="" type="checkbox"/> Cot	Protection classification: <input type="checkbox"/> I <input checked="" type="checkbox"/> II					
Bed type: WICKIE	Inventory Number:					
Location:	Serial Number:					
Grade level (IEC60601-2-52): <input type="checkbox"/> 1 <input type="checkbox"/> 2 <input type="checkbox"/> 3 <input type="checkbox"/> 4 <input type="checkbox"/> 5						
Manufacturer: Burmeier GmbH & Co. KG		User-specific: none				
Testing equipment used (Type / Inventory Number):		1.				
MPG Classification: Class I		2.				
I. Visual Inspection				OK	Not OK	Defect Description
Visual inspection of the electrical components						
Stickers and type plates	Presence, legible					
Housing of transformer unit / Power adapter	Damage					
Motor housing and lifting rods of motors	Correct position, Damage					
Hand switch	Damage, Foil intact					
Motor-, hand switch- and power cables	Damage, Routing					
Plug and plug covering strip of control unit	Available, Correct position					
Visual inspection of the mechanical components						
Stickers and type plates	Presence, legible					
Safe working load / patient weight	Presence, legible					
Bed frame	Damage, Deformation					
Spring wood slats	Damage, Splinters					
Castors	Damage					
Reclining surface	Damage, Deformation					
Wooden boards	Damage, Splinters					
Welded seams	Split welded seams					
Side rail	Damage, Splinters					
Knurled screws	Tight fit					
Wearing parts such as joints	Damage					
II. Electric measurements according to DIN EN 62353:2008-08 (leakage current, direct)						
Proceed as follow:				OK	Not OK	Defect Description
1. Plug the power adapter power cable into the test socket on the measuring instrument.						
2. Insert the plug of the 24 volt cable into the special adapter (extra accessory; request from BURMEIER).						
3. Connect the special adapter with the test probe of the test unit.						
4. Start procedures with the measuring instrument						
	Limit Value	Value of first measurement	Current actual value			
Result: Bed SK II (Type B)	0,1 mA	mA	mA			

III. Performance check		OK	Not OK	Defect Description
Performance check of the electrical components				
Motion limit switching of the motors	Automatic cut-off			
Hand switch, blocking functions	Test according to instructions for use			
Drive motors	Text, Abnormal Noise Development			
Strain relief and security of routing the cable	Secure, Correct position			
Strain relief for power adapter cable	Secure hold			
Performance check of the mechanical components				
Joints and pivots	Smooth operation			
Trapeze with belt strap	Secure hold			
Castors	Drive and brake			
Emergency lowering of the backrest	Test according to instructions for use			
Side rail	Safe locking / Secure hold / Safe unlocking			
Lower leg rest	Latching			
Accessories (e. g. trapeze pole, trapeze)	Secure, Damage, Suitability			
Result of the inspection:				
All values within permissible range: <input type="checkbox"/> yes <input type="checkbox"/> no		Next inspection date:		
Inspection passed: <input type="checkbox"/> yes <input type="checkbox"/> no				
If inspection was not passed: <input type="checkbox"/> Defective, do not use the bed! => Repair <input type="checkbox"/> Defective, do not use the bed! => Take out of Service <input type="checkbox"/> Bed does not meet the safety standards				
Test approval sticker applied: <input type="checkbox"/> yes <input type="checkbox"/> no				
Part of this inspection report:				
Comments:				
Inspected on:	Inspector:	Signature:		
Evaluated on:	Operator / Expert:	Signature:		

BURMEIER GmbH & Co KG Pivitsheider Straße 270 D - 32791 LAGE ☎ + 49 - 05232 - 9841 - 0		
Model	Wickie	
Type	123456	
Serie	11 - 2002 - 123456	
		
Model	Bettenmodell	
Type	Artikelnummer	
Serie	Seriennummer	
Model	Bed model	
Type	Item Number	
Serial	Serial Number	

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SPARE PARTS

The corresponding replacement parts can be obtained from BURMEIER by specifying the item number, the order number and the serial number. The necessary details are found on the type plate under the cross tubing of the reclining surface frame.

MANUFACTURER'S ADDRESS

In order to maintain operational safety and the right to claim under warranty, only original BURMEIER replacement parts may be used!

For ordering replacement parts, customer service requests and other questions, please contact:

Burmeier GmbH & Co. KG
(A subsidiary of the Stieglmeyer Group)

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REPLACEMENT OF ELECTRICAL COMPONENTS

Dangerous voltage, electric shock possible

- Before doing any work unplug the main power cable from the wall socket!
- Any work on the electrical equipment may only be performed by customer service, the motor manufacturer or qualified and authorized electricians under consideration of the applicable VDE rules and safety guidelines!

- Only dismantle the drive motors when the reclining surface is in the horizontal position, otherwise there is the risk of being injured by falling parts of the mattress base.

- All the drive components are maintenance-free and must not be opened. In the event of a malfunction, the corresponding components should always be replaced in full!

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Exchange of the Hand switch

- ☞ Unplug the power plug from the wall socket.
- ☞ Unplug the plug at the end of the spiral cable of the hand switch from the connector underneath the reclining surface.
- ☞ Lay the cable of the new hand switch so that it cannot be damaged by moving parts of the bed.
- ☞ Insert the plug at the end of the spiral cable of the new hand switch into the connector of the hand switch.
Take care that the O-ring on the plug is not damaged, otherwise the connection may not be moisture-proof.
- ☞ Carry out a function test of the electrical adjustments after the exchange.

Exchange of the power adapter

- ☞ To perform electrical tests the power adapter can be removed from the bed. The bed can remain occupied.
- ☞ Unplug the power cable from the wall socket.
- ☞ Unplug the plug of the 24-Volt connecting cable from the connecting socket at the head section.
- ☞ Connect the plug of the new power adapter into the connecting socket.
- ☞ Take care when laying the cable that it will not be damaged by moving parts or be run over.
- ☞ Carry out a function test of the electrical adjustments after the exchange.

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TROUBLESHOOTING

The following table on page 45 is a guide to rectifying common malfunctions:

If a function fault is detected during operation, which the user cannot rectify with the use of the trouble-shooting table aid, the specialized staff for maintenance and repair of the appropriate owner is to be called.

The user may **under no circumstances** try to eliminate defects at the electrical equipment!

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Dangerous voltage, electric shock possible!

Any work on the electrical equipment may only be performed by service personnel, the motor manufacturer or qualified and authorized electricians as required by the applicable VDE rules and safety guidelines!

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Trouble-shooting table aid

Problem	Possible causes	Solution
Hand switch / drive system not functioning	<ul style="list-style-type: none"> ■ Hand switch plug not properly connected ■ Transformer plug (24 volt) not properly connected ■ Hand switch or drive system defective ■ Functions are blocked on hand switch 	<ul style="list-style-type: none"> ☞ Test the plug connection ☞ Plug the transformer plug into the connection socket ☞ Inform your operator about necessary repairs ☞ Unlock functions (see page 23)
Green LED on power adapter not lit and drive systems are without function	<ul style="list-style-type: none"> ■ Power cable not properly connected ■ Wall socket without power ■ Power adapter defective 	<ul style="list-style-type: none"> ☞ Plug in the power cable properly ☞ Test the wall socket and fuse box ☞ Exchange the power adapter; inform your operator about necessary repairs
Green LED on power adapter lit continuously, yellow LED not lit when any hand switch button is pushed	<ul style="list-style-type: none"> ■ Hand switch is defective ■ Transformer plug (24 volt) not properly connected ■ Functions are locked on hand switch 	<ul style="list-style-type: none"> ☞ Exchange the hand switch ☞ Test the plug connection ☞ Unlock the functions (see page 23)
Drives operate in one direction only	<ul style="list-style-type: none"> ■ Hand switch or drive system is defective 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs
Drives stop suddenly after prolonged adjustment times	<ul style="list-style-type: none"> ■ Heat switch in power adapter has been activated 	<ul style="list-style-type: none"> ☞ Too short minimum pause (see page 22) ☞ Exchange the power adapter; inform your operator about necessary repairs

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ACCESSORIES

Only the Original BURMEIER accessories listed below may be used. If other accessories are used, we can not be liable for any resulting injuries, defects and dangers.

When using side rail, drip/I.V., etc. with electrically adjustable beds, please note:

Make sure that the arrangement of accessories does not produce any crush or shearing zones for the patient when the back and leg rests are adjusted. If this cannot be ensured, the user must safely prevent the patient from adjusting the back- and leg rest.

► To do so, attach the hand switch out of reach (e.g. at the foot end of the bed), or lock hand switch adjustments.

		ANFORDERUNGEN AN DIE MATRATZE	REQUIREMENTS OF THE MATTRESS
180x 90 cm	200 x 90 cm	Grundlegende Abmessungen:	Basic dimensions:
10 - 15 cm	10 - 15 cm	Länge x Breite	Length x Width
min. 35 kg/m ³	min. 35 kg/m ³	Dicke/Höhe	Thickness/Height
min. 4,2kPa	min. 4,2kPa	Raumgewicht des Schaumstoffs	Volume density of foamed material
		Stauchhärte	Compression load deflection
		Zu beachtende Normen:	Norms to be observed:
		DIN 13014	DIN 13014
		DIN 597 Teil 1 und 2	DIN 597 Section 1 and 2

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Hand switch with locking function

Type
Protection

Motors for reclining surface height adjustment

Type
Power / Size / Lift
Input Voltage
Operating Time
Protection

Motor for backrest

Type
Power / Size / Lift
Input Voltage
Operating Time
Protection

Motor for thigh rest

Type
Power / Size / Lift
Input Voltage
Operating Time
Protection

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Explanation of Icons:

♦ Unit with Type B application according to IEC 601-1 (Special protection against electric shock)

♦ Protection Class II device, double insulation

♦ Transformer with thermal fuse block

♦ Safety transformer in accordance with VDE 05510

♦ Caution! Consult instruction manual

♦ The electrical equipment offers protection from internal dust deposits and is splash-proof from all sides

♦ The electrical equipment is dust-proof from internal dust deposits and is splash-proof from all sides

♦ Conformity mark in accordance with Medical Products directive 93/42 EWG

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AMBIENT CONDITIONS

Noise level during motor adjustments

The ambient conditions stated below must be observed:

In storage:

Storage temperature

Relative humidity

In operation:

Ambient temperature

Relative humidity

Air pressure

APPLIED STANDARDS AND GUIDELINES

Medical Products Directive

Repeat testing ... of medical electrical units

Technical aids for handicapped persons

Handicapped friendly construction

Risk analysis for medical products

Adjustable beds for handicapped persons

Safety for electro-medical equipment

Electromagnetic tolerances

Medical electrical equipment –

Particular requirements for medical beds

(reduced to applications for in home care)

Children's cots and folding cots for domestic use

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DISPOSAL INSTRUCTIONS

- ◆ This cot is - if electrically adjustable - classified as a commercially used electric appliance (b2b) in accordance with WEEE-Guidelines 2002/96/EG (Electronics Law).
- ◆ Exchanged electronic components (drives, control unit, hand switch, etc.) of these cots are considered electronic scrap and must be disposed of according to the WEEE-Guidelines.
- ◆ With cots that came into use after August 13, 2005 (see designation "Serial" on the type label at the head of the bed), the operator is required to not dispose of the electrical components in the municipal waste area, but to send them directly to the manufacturer. BURMEIER and their service and sales partners will take these parts back.
- ◆ Our general terms and conditions apply for this redemption.
- ◆ No longer useable rechargeable batteries have to be disposed of according to regulations and do not belong in the house garbage.
- ◆ With all disposed components the owner must make sure that they are not infectious or contaminated.
- ◆ In the event that the cot must be scrapped, the plastic and metal parts used must be disposed of separately, following the proper disposal procedure.
- ◆ Pneumatic springs are under high pressure! Before disposal they have to be depressurized. Please contact the manufacturer of the device on how to depressurize them (see type plate).
- ◆ Hydraulic units are made up of mostly metal and plastic. Before disposing according to manufacturer guidelines the oil has to be drained and disposed of properly.
- ◆ Contact your local community, disposal facility or our service department with any questions.

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We,

Burmeier GmbH & Co. KG
Pivitsheider Strasse 270
32791 Lage / Lippe
Germany

hereby declare that the product named below:

Cot WICKIE

complies with the regulations of the EC Directive 93/42/EEC for Medical Products and the additional safety requirements of the supreme state authority from 22 May 2001 for cots.

Applied Standards:

- DIN EN 62353: 2008-08 Repeat inspections ... of electromedical equipment
- DIN EN ISO 14971:2001-03 Medical devices – Application of risk management to medical devices
- EN 12182: 1999-11 Technical aids for disabled people
- DIN 32977-1:1992-07 Adapted designing for disabled people
- DIN-EN 716-2:2008 Children's cots and folding cots for domestic use
Part 2: Test methods
- DIN 32623:2009 Hospital children's cots made from metal and plastic- Safety requirements and testing
- IEC 60601-2-52: 2009-12 Medical electrical equipment - Particular requirements for the basic safety and essential performance of medical beds

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No part of this manual may be reproduced with-
out the prior written permission of the publisher!

All rights reserved.
Subject to technical changes!

Stand: 2011-10-24

Updated: 2011-10-24